



Ride Custom Systems Shape Capture Check List

This comprehensive tools and materials check-list will ensure you have all materials needed for capturing the shape of your client for a Ride Custom Cushion and/or Back support.

- Order Forms (download order forms at www.ridedesigns.com/orderforms.html)
- Tape Measure
- Camera

Ride Custom Cushion tools:

- Ride Custom Evaluator Cushion of appropriate size for Evaluator Based cushions (these are the blue covered cushions that came in your Ride Custom Cushion Starter Kit. Make sure Ride CAM® Wedges are in the front pouch of the Evaluator cover.):
 - Small (14" W x 14" D)
 - Medium (16" W x 16" D)
 - Large (16" W x 16" D)

Note: Evaluator cushions are used to determine if a standard off-loading contour will safely and correctly support a person. If successful, build a Ride Custom Cushion around this contour by filling out the order form and fax to Ride Designs. If unsuccessful, then proceed to capture the person's shape in the impression foam simulator.

- Box of Ride Gauges
- Ride Custom Cushion impression foam simulator of appropriate size for Simulator Based cushions:
 - Small for hip widths less than 14"
 - Medium for hip widths less than or equal to 17"
 - Large for hip widths greater than or equal to 18"
- Box of simulator refresh kits, same size as simulator selected.
- 2" hook Velcro to attach simulator/evaluator to wheelchair.

Ride Custom Back tools:

- Back kit:
 - Inside your original kit you will find everything needed:
 - Inner spandex bag with outer plastic bag installed.
 - Vacuum pump
 - Additional outer replacement bags
 - A zip-loc bag with electrical tape, shipping label, order forms, and indelible pencil (make sure it is sharpened).
 - M-L measuring stick
- 1 Box of splinting material and 2 rolls of plaster for each shape to be captured.
 - Note: A pre-paid shipping label, new outer bag, pencil, splinting material and 2 plaster rolls accompany every Ride Custom Back you order.
- Roll of high quality masking tape for quick patch of the occasional bag tear.
- Bucket for plastering
- Examination gloves for use during plastering.
- Tarp to protect floor.
- Scissors for removing used outer bags.



Ride Designs®
a branch of Aspen Seating, LLC

4211-G South Natches Court
Sheridan, Colorado 80110

toll-free 866.781.1633
phone 303.781.1633
fax 303.781.1722

Face Sheet



Ride Custom Systems Face Sheet

(Please fill in one face sheet per client order)

Client First Name / Last Initial _____

Components ordered:

Please attach appropriate order form for each component

- Ride Custom Cushion**
 - Simulator (shape capture) based,
Simulator number: _____
 - Evaluator based
- Ride Custom Back**

Account #: _____

PO # _____

Date _____

1. General Information

Supplier _____

Contact Name _____

Address _____

City _____ State _____ Zip _____

Phone _____ E-mail _____

Ship To (if different from above)

NOTE: Ride Custom Systems must be fitted by a Ride Certified provider, and WILL NOT be drop shipped to end users.

Address _____

City _____ State _____ Zip _____

Phone _____ E-mail _____

Referral Source

Facility _____ Clinician Name _____

Phone # _____ E-mail _____

2. Client Information

WARNING: Caution should be exercised when capturing shapes in Ride Simulators for people with osteoporosis, bone cancer, history of pathological fracture, osteogenesis imperfecta, or any brittle bone condition.

First Name/Last Initial _____ M F

Diagnosis _____ Height _____ Weight _____

Hip width: _____" Leg length: Left _____" Right _____"

3. Mobility Base Specifications

Wheelchair Make _____ Model _____

Frame Width _____" Depth _____"

NOTE: Ride Designs does not recommend use of Ride Custom Cushions on wheelchairs with **ERGO FRAMES**.

4. Please fill in and attach the appropriate order form for each Ride Custom Component ordered.



Ride Designs®
a branch of Aspen Seating, LLC

4211-G South Natches Court
Sheridan, Colorado 80110

toll-free 866.781.1633
phone 303.781.1633
fax 303.781.1722

