



Ride Corbac Push Handle Mounting Accessory (CB-PHMA) Installation Instructions

- Follow these supplemental instructions should you desire to **replace existing sling back upholstery on a wheelchair that has back canes with integrated push handles.**

This kit includes the following:

- Support Strap Assembly
- (2) 1" diameter cane clamps
- (2) 3/4" diameter cane clamps with rubber inserts

WARNING: This supplement does NOT replace the Corbac owner's manual. Safe application of this information depends on its use in conjunction with the Corbac owner's manual. Failure to heed the instructions contained in the Corbac owner's manual could result in serious injury.

Perform the following steps to install your Mounting Accessory:

1. Verify that the Corbac will be used without the wheelchair back upholstery and that the rear canes are equipped with push handles.
2. Determine the cane diameter at the location where the top of the Corbac height will be positioned.
3. The Mounting Accessory is pre-assembled using the clamps for a 1" cane diameter. If the Corbac is to be used with 3/4" diameter canes, use the 3/4" clamps provided. Replace one side at a time using the pre-assembled side as an example.
4. To install the Mounting Accessory, remove the screws from the (2) clamps and spread the clamps around the back canes at the desired back height.
NOTE: The 3/4" diameter clamp requires a rubber insert (included) to be used with each clamp. The 1" clamp does not require rubber inserts.
5. Refer to the "TOP FRONT" label for proper orientation. Front oriented toward sitter.
6. Tighten the screws securely.



WARNING: Failure to configure and tighten the assembly as directed could result in failure of the Mounting Accessory, and could lead to serious injury.

7. The Mounting Accessory can be used on wheelchairs up to 18" in width. The webbing can be adjusted in length. Ensure that the webbing is doubled back through the friction device, as it is provided, to prevent slipping.
8. Tighten the strap sufficiently to support the top of the Corbac. Some slack will be required to permit trunk extension between the back canes once the Corbac is installed. **NOTE:** Excessive slack may compromise the kit's ability to provide adequate support to the installed Corbac.
9. After setting strap length, re-adjust clamps so the triangular metal brackets are in line with the tension strap. Tighten the screws securely.
10. Install the Corbac over the Mounting Accessory following steps 2 through 6 in the Ride Corbac Owners Manual.

