



Ride Designs Parts Manual

Ride Designs®
a branch of Aspen Seating, LLC

8100 SouthPark Way, C400
Littleton, Colorado 80120 USA

toll-free: 866.781.1633
phone: 303.781.1633
www.ridedesigns.com

Table of Contents


Content	Page
Ride® Custom Back Parts	
FlexLoc Assembly	4
FlexLoc Parts	5
FlexLoc Fixed Pin Assembly	6
FlexLoc Quick Release Pin Assembly	7
FlexLoc Universal Clamp Assembly	8
FlexLoc Adapter Plate Assembly	9
FlexLoc Power Screw Kit	10
Invacare T-Nut Adapter Kit	11
Ride Custom Back Headrest Adapter Kit	12
Ride Custom Back Mounting Assembly	13
Ride Custom Back Double Mounting Assembly	14
Ride® Java® Back Parts	
Java Back Mounting Assembly, Left	16
Java Back Mounting Assembly, Right	17
Java Decaf Back Assembly	18
15-20" Java Back Latch Pin Bracket Assembly	19
Java Grande Back Latch Pin Bracket Assembly	20
Pelvic Wedges (Java TLP, Corbac TLP)	21
Java Back Headrest Adapter Kit	22
Java Back Cover Assembly	23
Java Back Plastizote	24
Java Back Polyurethane Foam	25
Java Back Cover	26
Java Back Fixed Lateral Assembly	27
Java Back Swing Away Lateral Assembly, Right	29
Java Back Swing Away Lateral Assembly, Left	31
Ride® Java® Cushion Parts	
Java Cushion Assembly	34
Java Cushion Top Foam	35
Java Cushion Base	36

Ride[®] Custom Back Parts



FlexLoc[®] Assembly

Note: When ordering a FlexLoc Assembly two part #'s will need to be ordered:

- 1) Choose the part # for the size of FlexLoc needed with the appropriate Quick Release or Fixed Mount suffix.
- 2) Choose the part # for either Clamps or Adapter Plate. The price of the Clamp or Adapter Plate is included in the FlexLoc Assembly price and will show up as \$0.00 on the quote or invoice.

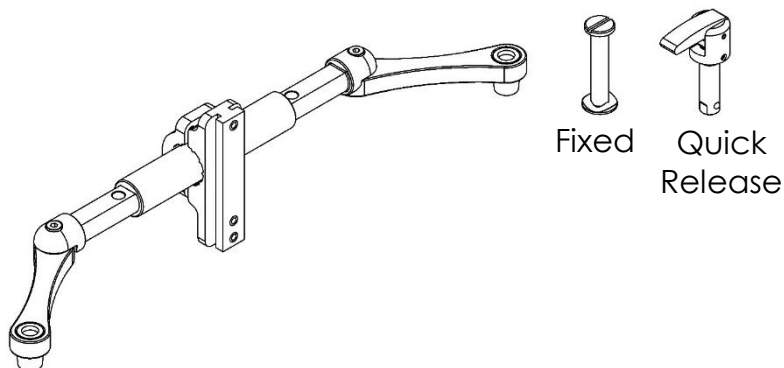


Part No.	Description
FLXLOC-MS-FM	FlexLoc, Small (10" - 13") w/ Fixed Mount
FLXLOC-MS-QR	FlexLoc, Small (10" - 13") w/ Quick Release
FLXLOC-MM-FM	FlexLoc, Medium (14" - 17") w/ Fixed Mount
FLXLOC-MM-QR	FlexLoc, Medium (14" - 17") w/ Quick Release
FLXLOC-ML-FM	FlexLoc, Large (18" - 21") w/ Fixed Mount
FLXLOC-ML-QR	FlexLoc, Large (18" - 21") w/ Quick Release
FLXLOC-MX-FM	FlexLoc, Extra Large (22" - 24") w/ Fixed Mount
FLXLOC-MX-QR	FlexLoc, Extra Large (22" - 24") w/ Quick Release

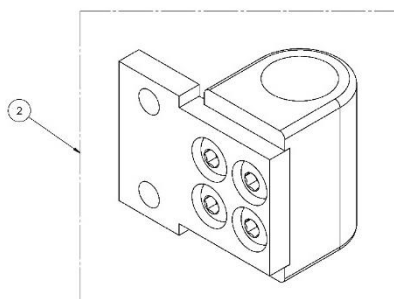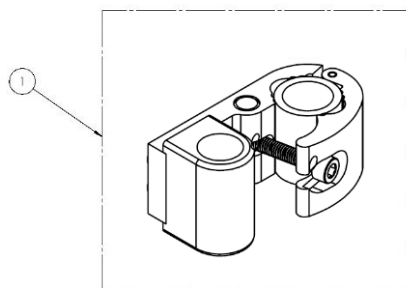


Item	Part No.	Description
1	FL-MCI	FlexLoc Clamp Set, (Pair), Installed
2	FL-MCI-P1	FlexLoc Adapter Plate Set, (Pair), Installed


FlexLoc® Parts

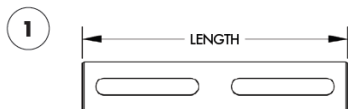


Part No.	Description	
025-146	Small Connecting Tube	4-1/4"
025-126	Medium Connecting Tube	5-3/8"
025-147	Large/XL Connecting Tube	9-3/8"

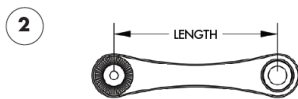

Part No.	Description	
6000-178	Small Adjustment Arm, Pair	2-1/4"
6000-179	Medium/Large Adjustment Arm, Pair	3-3/4"
6000-180	X-Large Adjustment Arm, Pair	5"

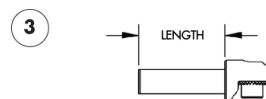


Part No.	Description	
6000-181	Small Extension Arm	2"
6000-140	Large Extension Arm	2-5/8"

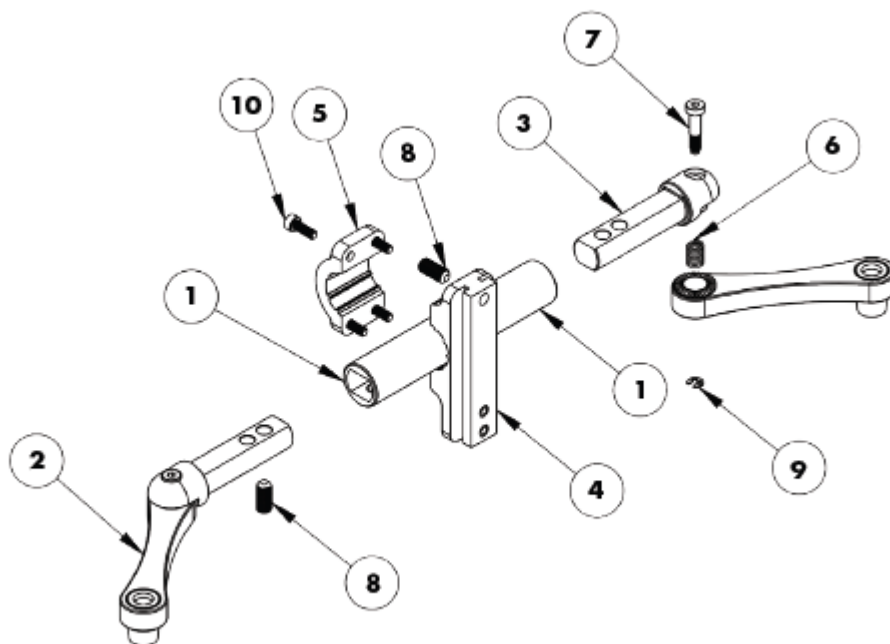


Item	Part No.	Description
1	See Chart	Connecting Tube
2	See Chart	Adjustment Arm Assembly, Pair
3	See Chart	Extension Arm Assembly, Pair
4	025-139	1.00 Diameter Clamp Body
5	025-138	1.00 Diameter Clamp Hinge
6	025-140	.36 O.D. x .055 Wire Diameter x 1.00
7	015-370	Modified M6 x 27mm Low Profile SHCS
8	015-292	M8 x 18mm Set Screw w/ Nyloc Patch
9	015-344	3/16" E-Clip Retaining Ring
10	015-295	M5 x 14mm Socket Head Black Zinc

FlexLoc[®] Fixed Pin Assembly

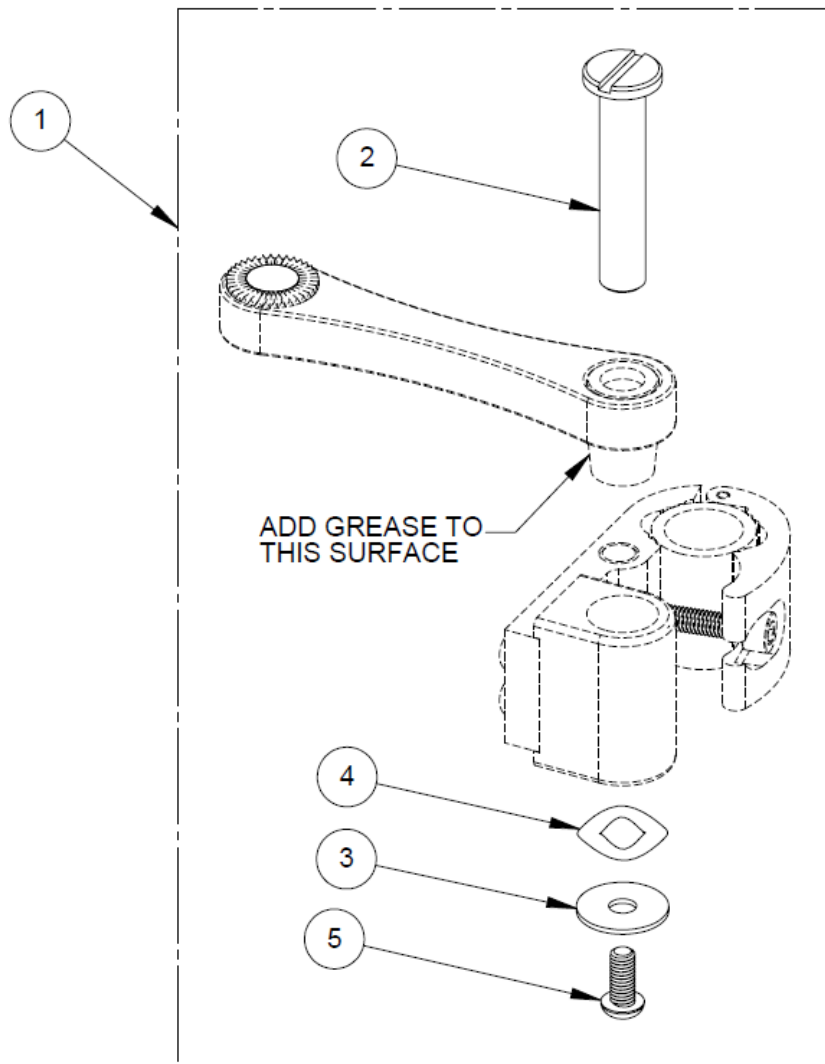


Item	Part No.	Description
1A	6000-145	FlexLoc Fixed Pin Assembly, Each
1B	FL-FM	FlexLoc Fixed Mount Pin Assembly, Pair
2	015-303	.375 Clevis Pin
3	015-354	17/64" I.D. x .875 O.D. Large Diameter Washer, Black
4	015-308	.44 I.D. x .86 O.D. Triple Wave Washer
5	015-317	M6 - 1.0 x 14mm Button HD, Black-Zinc W/ Nylock Patch

FlexLoc[®] Quick Release Pin Assembly

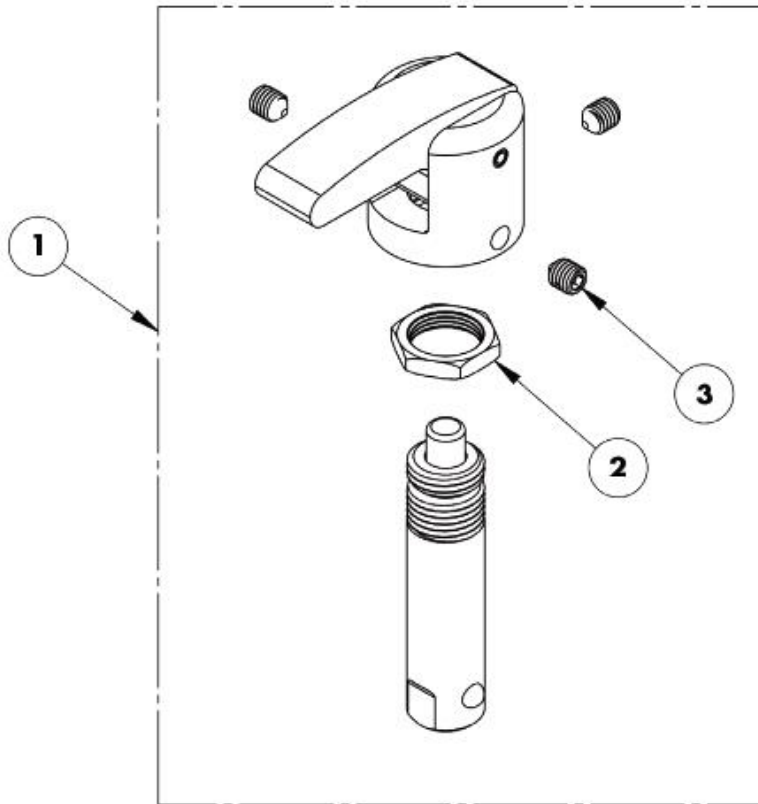


Item	Part No.	Description
1A	6000-224	Quad Release Assembly, Each
1B	FL-QR-RK	FlexLoc Quick Release Pin Retrofit Kit, Pair
2	015-332	Panel Nut, 1/2-20 UNC, 1/8" Thick
3	015-305	M5 x 6mm Cone Point Set Screw

FlexLoc® Universal Clamp Assembly

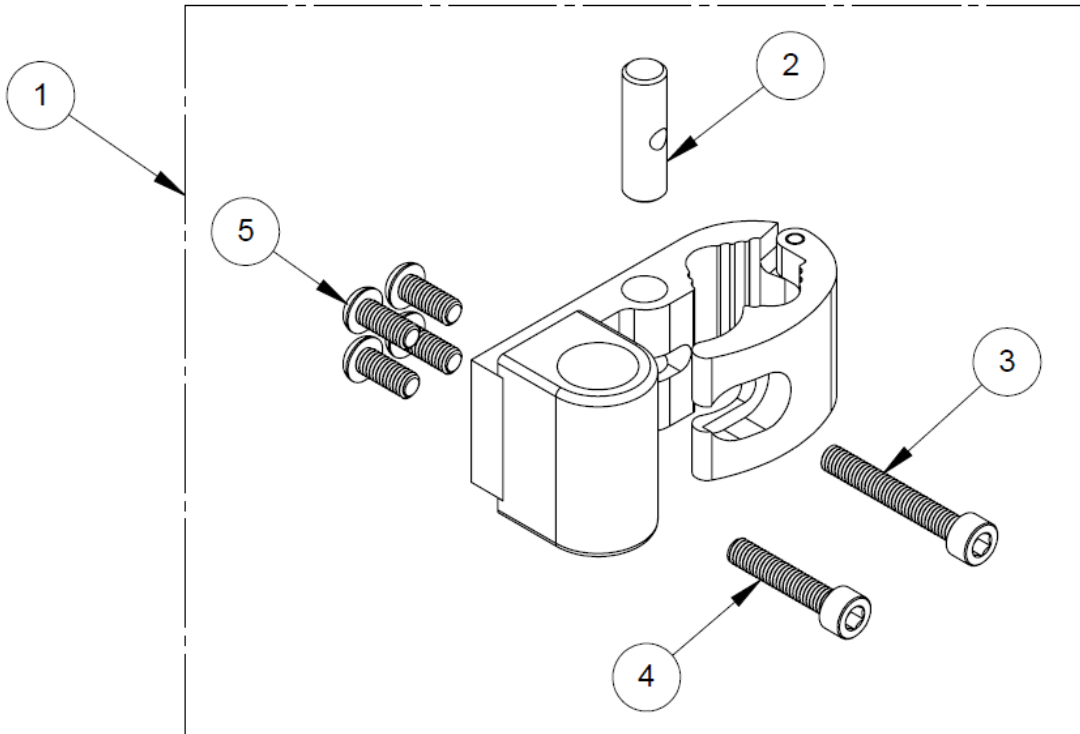


Item	Part No.	Description
1A	6000-420	Universal Clamp Assembly, Each
1B	FL-MC	Universal Clamp Assembly, Pair
2	025-135	.375 Diameter x 1.25 Long Barrel Nut
3	015-409	M6 x 40mm Socket Head Screw (Use with 1" Canes)
4	015-410	M6 x 30mm Socket Head Screw (Use with 3/4" Canes)
5	015-293	M6 x 14mm Button Head w/ Patch Black-Zinc

FlexLoc® Adapter Plate Assembly

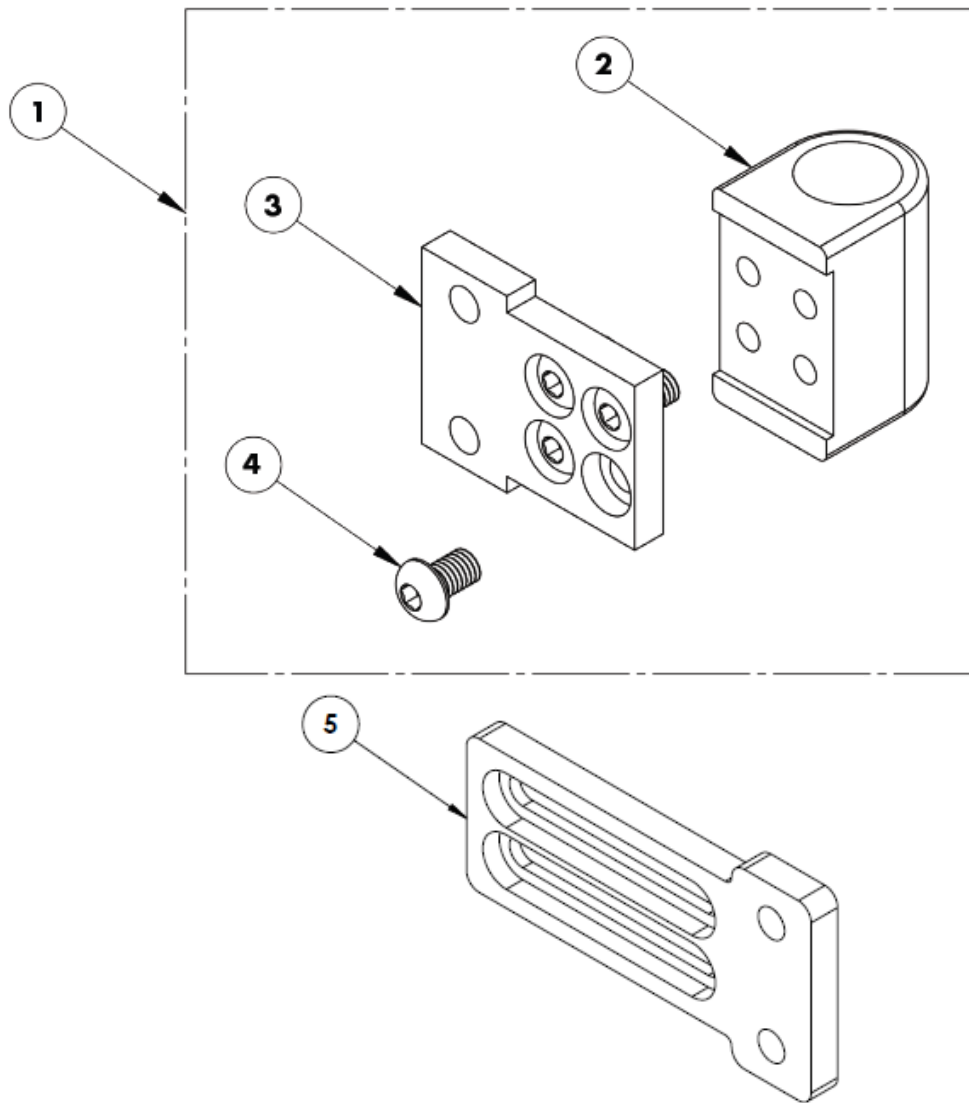


Item	Part No.	Description
1A	6000-187	FlexLoc Adapter Plate Assembly, Each
1B	FL-MC-P1	FlexLoc Adapter Plate Assembly, Pair
2	025-127	Receiver Block
3	025-154	FlexLoc Adapter Plate
4	015-334	M6 x 1.0 x 10mm Button Head w/ Patch Black Zinc
5	025-209	FlexLoc Adaptor Plate, Long

FlexLoc[®] Power Screw Kit

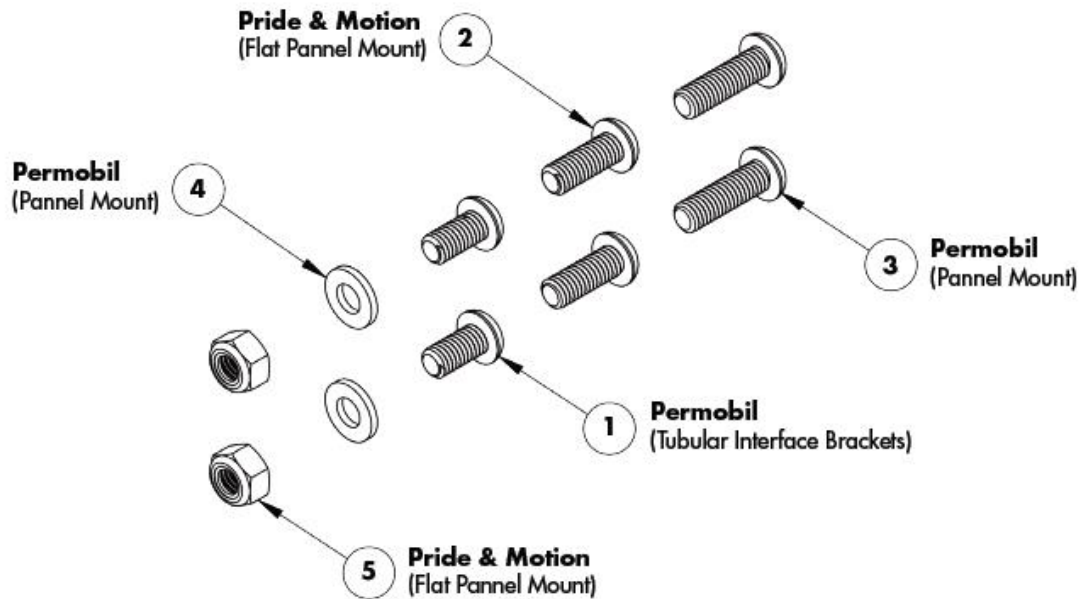


Item	Part No.	Description
1	015-333	M6 x 1.0 Diameter x 12mm Button Head w/ Patch Black Zinc
2	015-336	M6 x 1.0 Diameter x 16mm Button Head w/ Patch Black Zinc
3	015-327	M6 x 1.0 Diameter x 20mm Button Head w/ Patch Black Zinc
4	015-279	M6 x 1.0 Thick x 12mm O.D. Flat Washer Pack
5	015-337	M6 x 1.0 Lock Nut

Invacare® T-Nut Adapter Kit

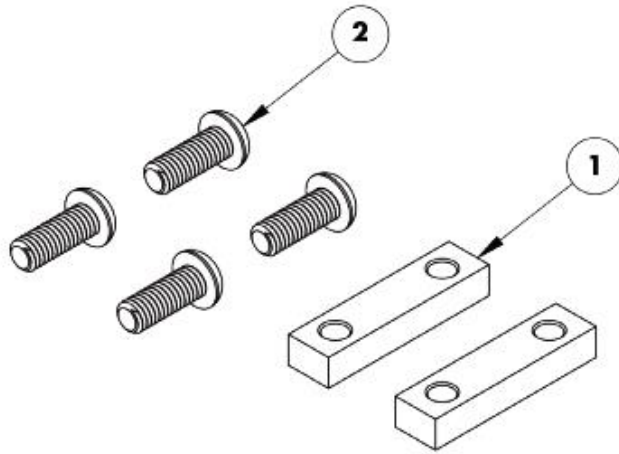


Item	Part No.	Description
1	025-172	M6 Double T - Nut
2	015-336	M6 x 1.0 Diameter x 16mm Button Head w/ Patch Black Zinc

Ride® Custom Back Headrest Adapter Kit

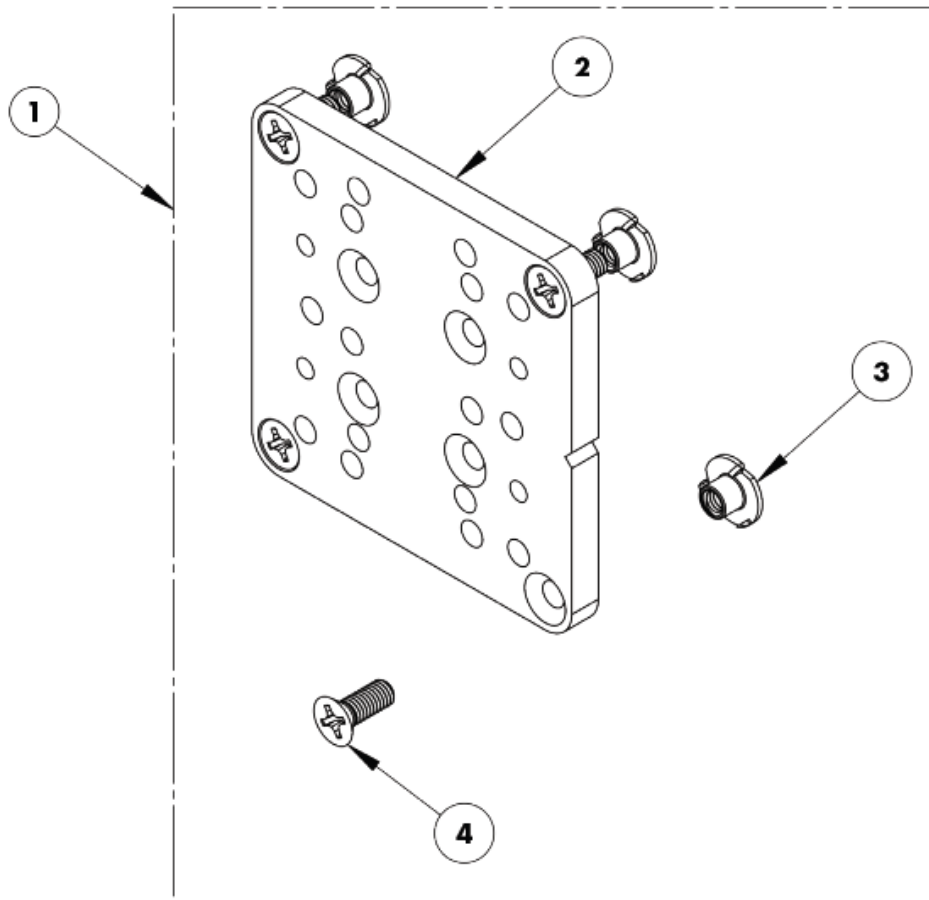


Item	Part No.	Description
1	RCB-UHMP-RK	Ride Custom Back, Universal Headrest Mounting Plate, Replacement Kit
2	025-073	Headrest Adapter Plate
3	015-036	T - Nut, 10-32
4	015-049	Screw, Flat Head, Phillips, 10-32 x .50

Ride® Custom Back Mounting Assembly

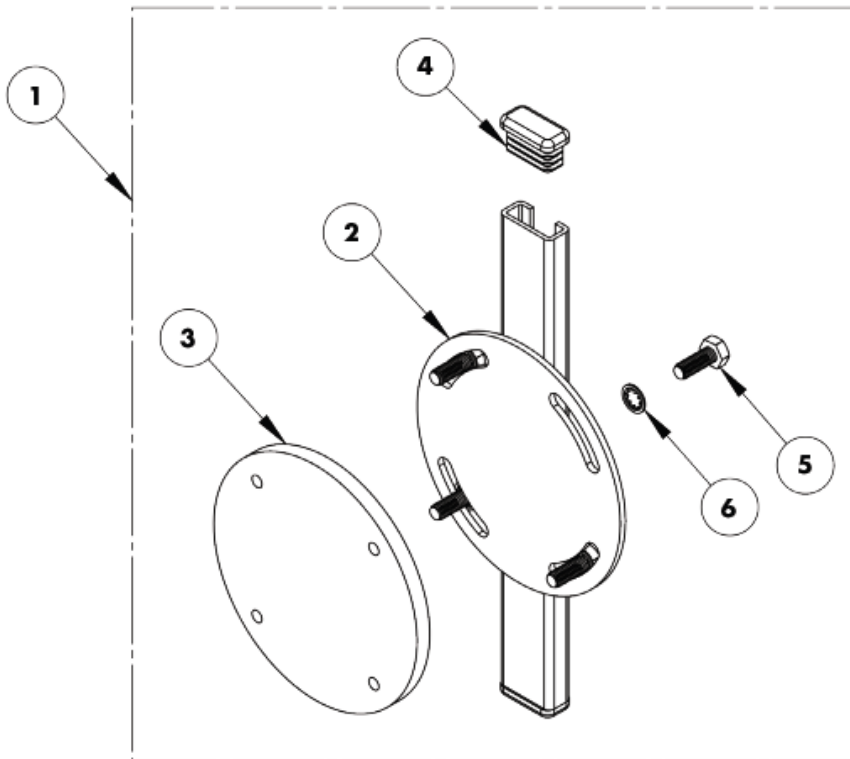

Item	Part No.	Description
1	FLXLOC-BKMT-S	RCB Single Mounting Assembly
2	025-032	RCB Mounting Assembly (Single)
3	025-031	RCB Mounting Back Plate
4	065-017	End Cap, Plastic
5	015-111	Screw, Hex Head, M6 x 16mm, Black, Nylon Patch
6	015-112	Lock Washer, Internal Tooth, M6


Ride® Custom Back Double Mounting Assembly

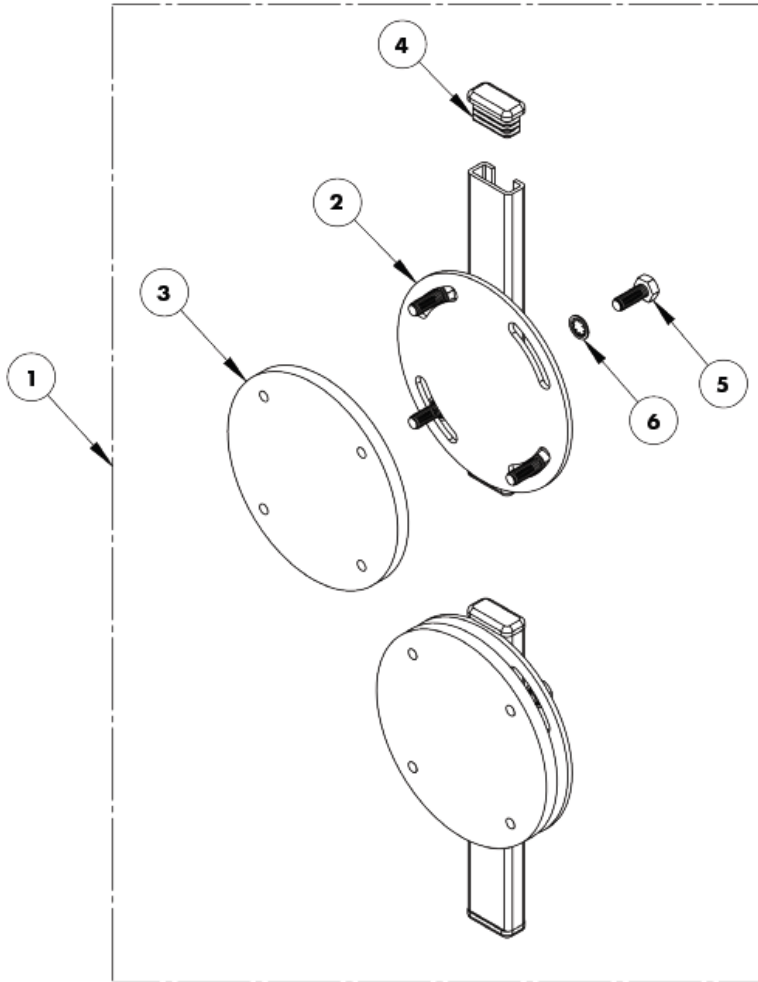

Item	Part No.	Description
1	FLXLOC-BKMT-D	RCB Double Mounting Assembly, Pair
2	025-059	RCB Mounting Assembly (Double)
3	025-031	RCB Mounting Back Plate
4	065-017	End Cap, Plastic
5	015-111	Screw, Hex Head, M6 x 16mm, Black, Nylon Patch
6	015-112	Lock Washer, Internal Tooth, M6


Ride[®] Java[®] Back Parts

Java® Back Mounting Assembly, Left

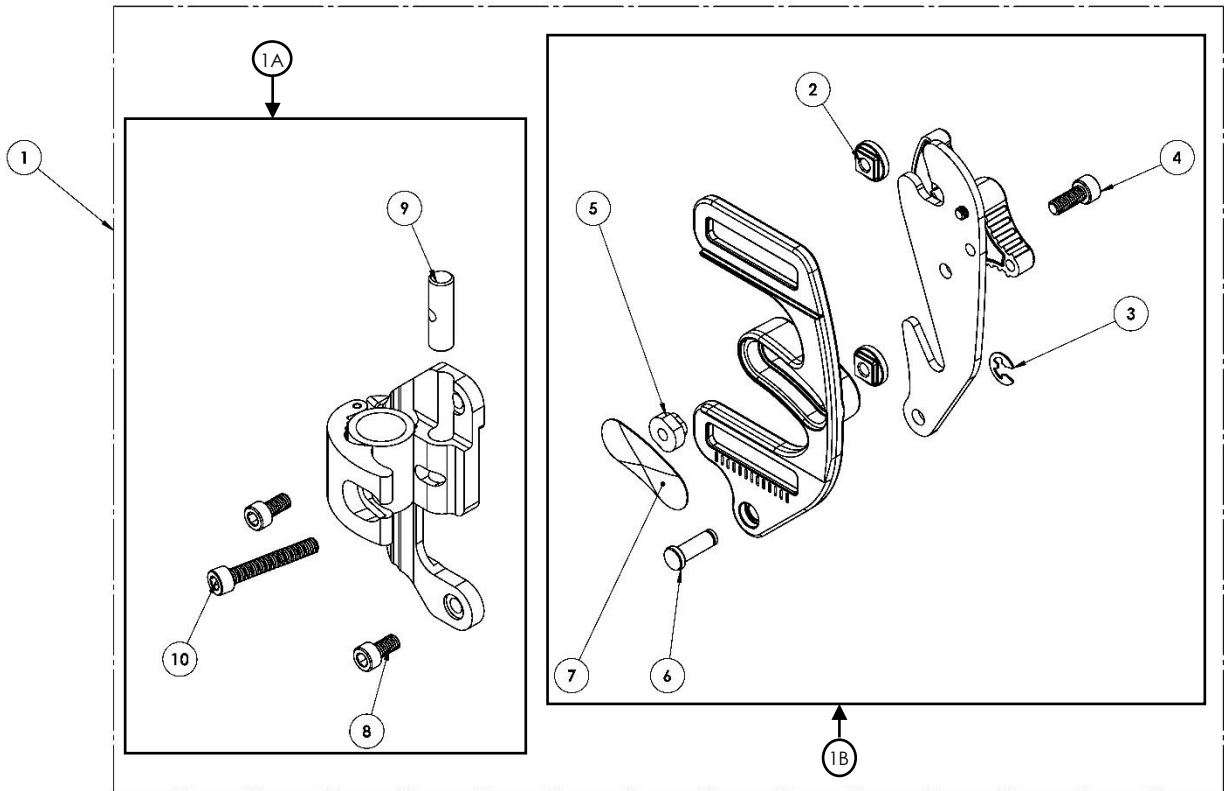


Item	Part No.	Description
1	JB-RK	Java Back GEN 2 Hardware Replacement Kit (Pair)
1A	6000-219	Java Back Clamps – Gen 2 (Pair)
1B	6000-651	Java Back Latch Assembly – Gen 2 (Pair)
2	025-335	M6 Slot Nut
3	015-276	1/4" E-Clip
4	015-220	M6 - 1.0 x 15mm Socket Head Screw w/ Nylock Patch (Black)
5	025-317	M6 Angle Slot Nut
6	015-252	1/4" x 3/4" Clevis Pin
7	025-319	Slot Cover
8	015-245	M6 - 1.0 x 12mm Socket Head Screw w/ Nylock Patch (Black)
9	025-135	.375 Diameter x 1.25 Long Barrel Nut
10	015-409	M6 - 1.0 X 40mm Socket Head Cap Screw, Fully Threaded

Java[®] Back Mounting Assembly, Right

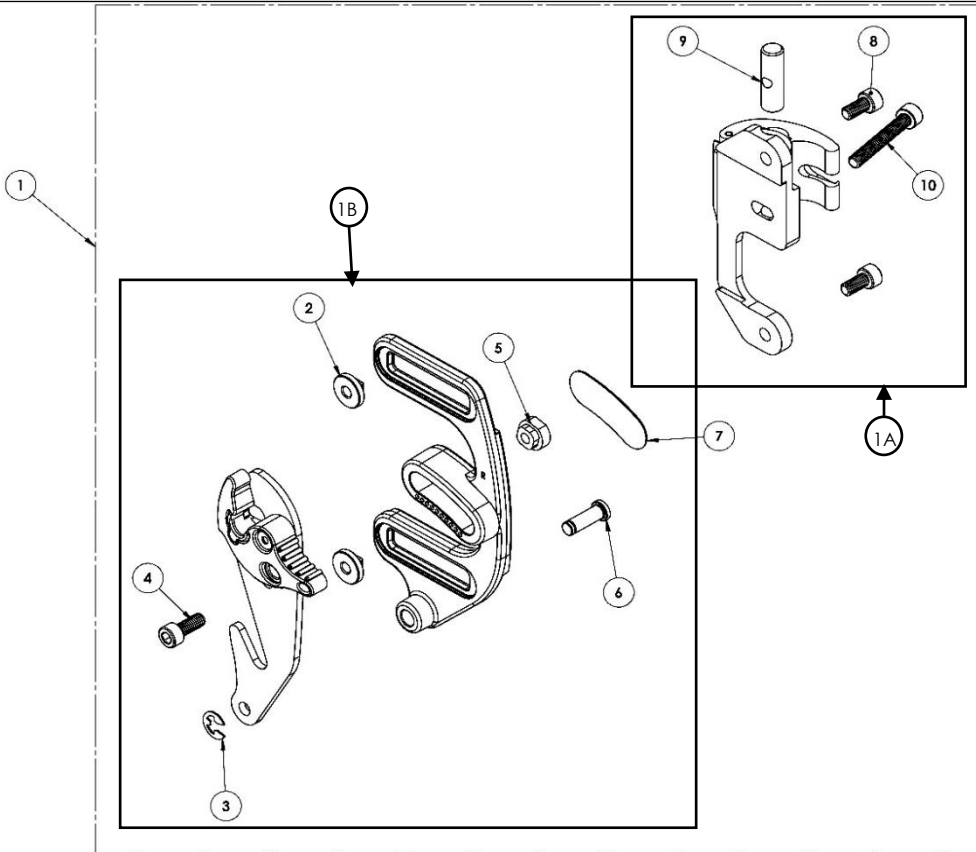


Item	Part No.	Description
1	JB-RK	Java Back GEN 2 Hardware Replacement Kit (Pair)
1A	6000-219	Java Back Clamps – Gen 2 (Pair)
1B	6000-651	Java Back Latch Assembly – Gen 2 (Pair)
2	025-335	M6 Slot Nut
3	015-276	1/4" E-Clip
4	015-220	M6 - 1.0 x 15mm Socket Head Screw w/ Nylock Patch (Black)
5	025-317	M6 Angle Slot Nut
6	015-252	1/4" x 3/4" Clevis Pin
7	025-319	Slot Cover
8	015-245	M6 - 1.0 x 12mm Socket Head Screw w/ Nylock Patch (Black)
9	025-135	.375 Diameter x 1.25 Long Barrel Nut
10	015-409	M6 - 1.0 X 40mm Socket Head Cap Screw, Fully Threaded

Java® Decaf Back Assembly

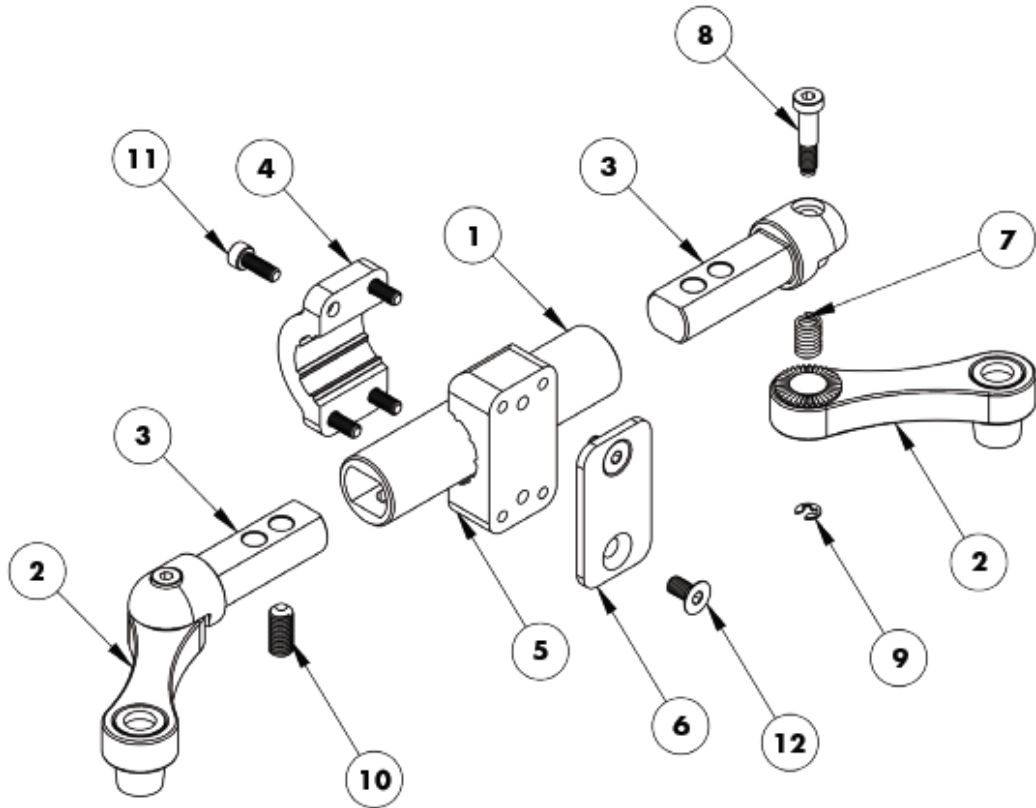


Item	Part No.	Description
1	025-146	Connecting Tube, Small
2	6000-178	Adjustment Arm Assembly, Small, Pair
3	6000-181	Extension Arm Assembly, Small
4	025-138	1.00 Diameter Clamp Hinge
5	025-161	1.00 Diameter Pediatric Clamp Body
6	025-162	Clamping Plate
7	025-140	.36 O.D. x .055 Wire Diameter x 1.00
8	015-370	Modified M6 x 27mm Low Profile SHCS
9	015-344	3/16" E - Clip Retaining Ring
10	015-292	M8 x 18mm Set Screw w/ Nyloc Patch
11	015-295	M5 x 14mm Socket Head Black Zinc
12	015-349	M6 - 1.0 x 12mm Flat Head Socket Black Zinc w/ Patch

15-20" Java® Back Latch Pin Bracket Assembly

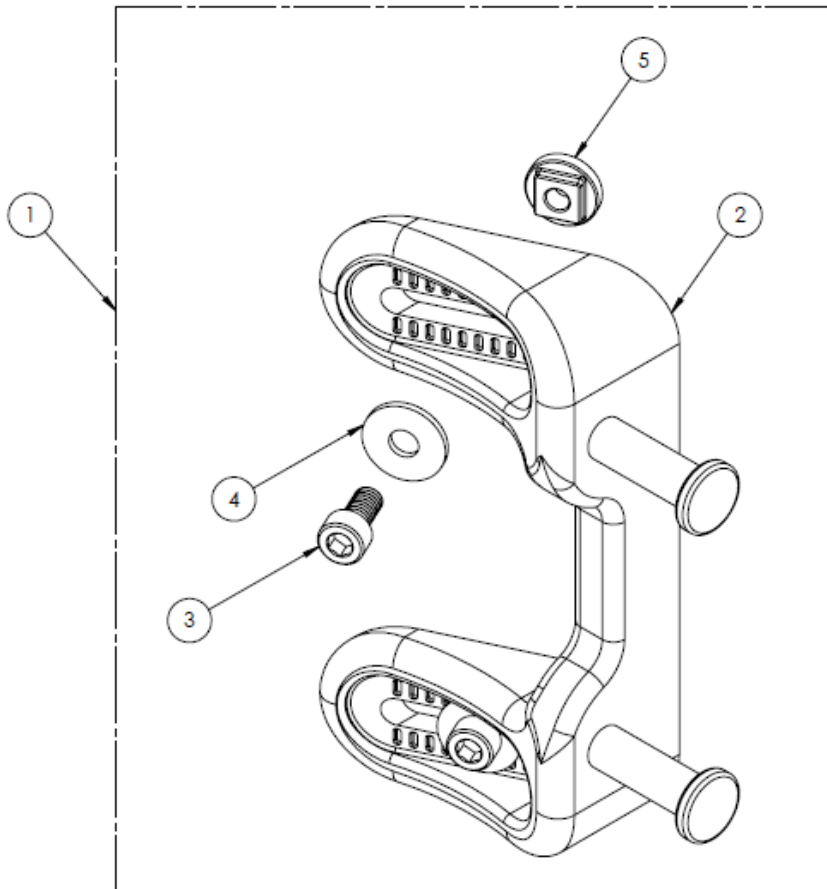


Item	Part No.	Description
1	JB-LP	Java Back Latch Pin Bracket, Regular and Tall Height (Each)
2	065-198	Latch Pin Bracket, Double Flanged Pins, Large
3	015-429	M6 - 1.0 x 14mm Socket Head Screw w/ Nyloc Patch (Black)
4	015-246	M6 x 18mm O.D. x 1.5mm Thick Flat Washer
5	025-335	M6 Slot Nut

Java® Grande Back Latch Pin Bracket Assembly

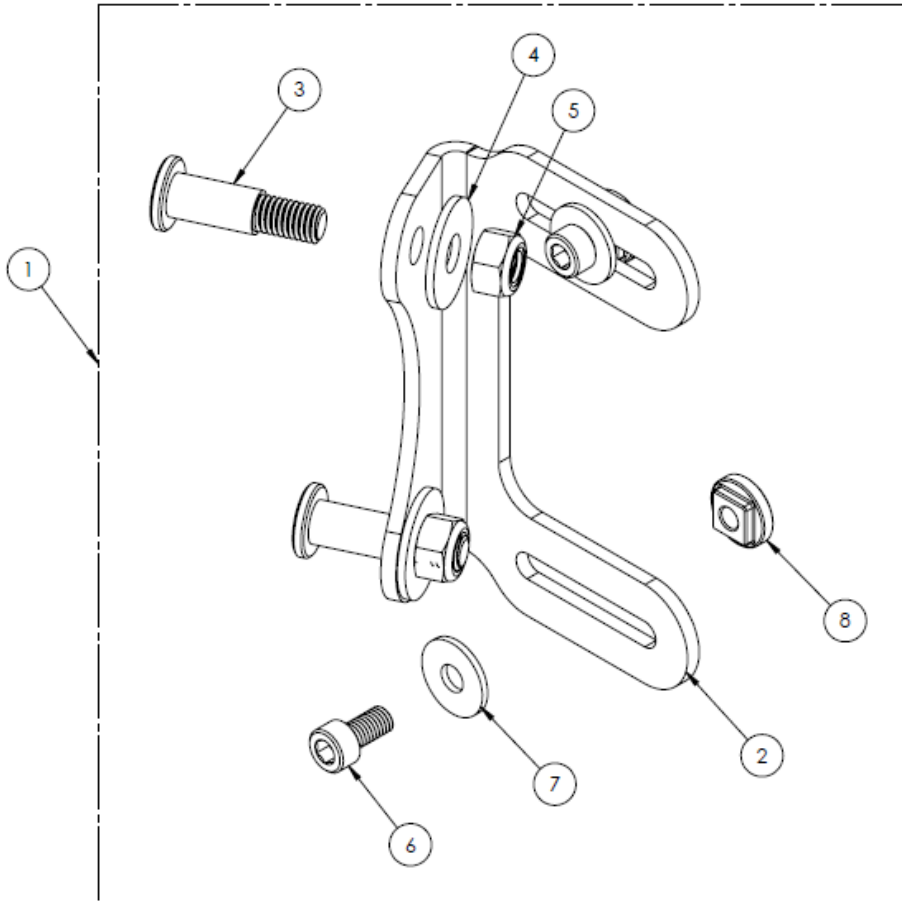


Item	Part No.	Description
1	JB-LPG	Java Back Latch Pin Bracket, Grande (Each)
2	025-213	Latch Pin Bracket, Large
3	025-214	Latch Pin, Java Bracket, Grande
4	015-381	M8 x 24mm O.D. Washer, Black
5	015-380	M8 x 1.25 Nylock Nut, Black
6	015-245	M6 - 1.0 x 12mm Socket Head Screw w/ Nylock Patch (Black)
7	015-246	M6 x 18mm O.D. x 1.5mm Thick Flat Washer
8	025-335	M6 Slot Nut

Pelvic Wedges

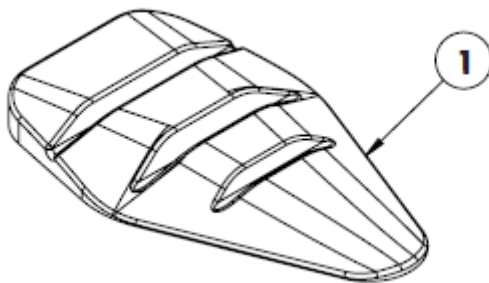


Item	Part No.	Description
1	JB-PW	Java Back TLP, Pair

Java[®] Back Headrest Adapter Kit

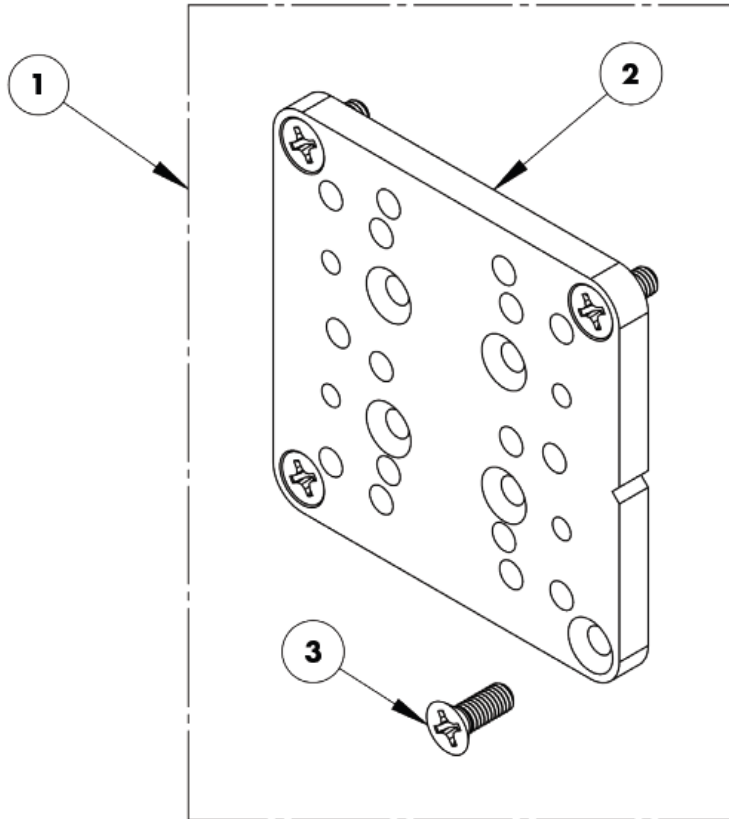


Item	Part No.	Description
1	JB-UHMP	Java Back, Universal Headrest Mounting Plate
2	025-073	Headrest Adapter Plate
3	015-049	Screw, Flat Head, Phillips, 10-32 x .50

Java® Back Cover Assembly

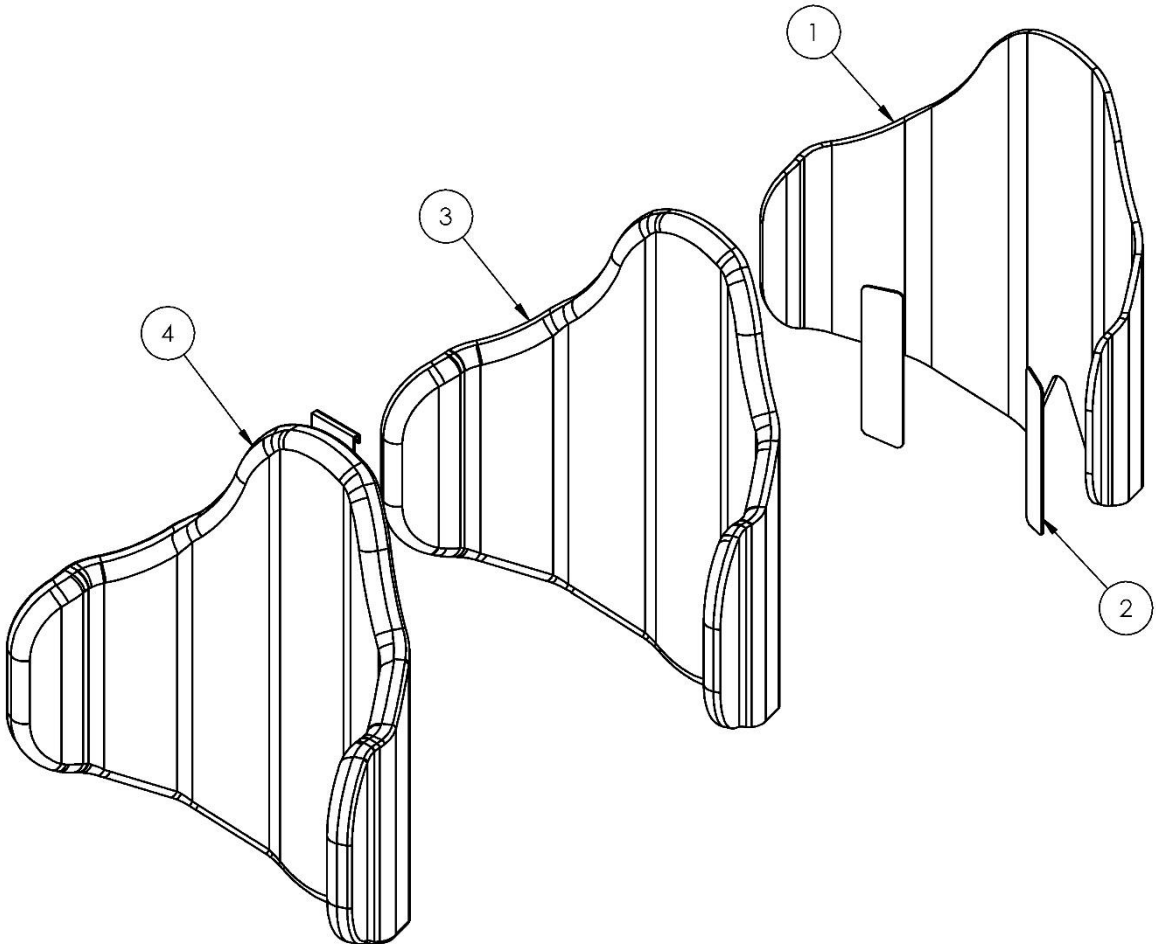


Item	Part No.	Description
1	See Chart	Plastizote
2	JB-FAS	Java Back Stay (Pair)
3	See Chart	Polyurethane Foam
4	See Chart	Java Back Cover

Java® Back Plastizote

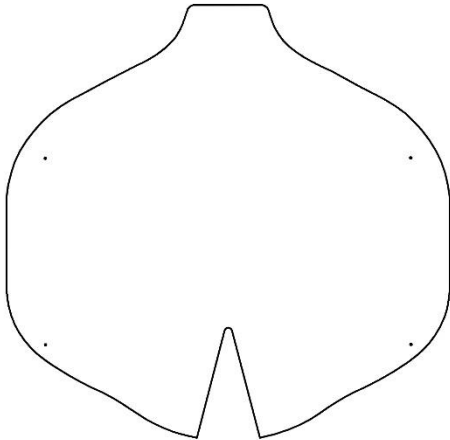


Part No.	Description
065-092	Plastizote, 1/4", 12" (Decaf)
065-093	Plastizote, 1/4", 12" Tall (Decaf)
065-094	Plastizote, 1/4", 13" (Decaf)
065-095	Plastizote, 1/4", 13" Tall (Decaf)
065-228	Plastizote, 1/4", 14"
065-230	Plastizote, 1/4", 14" Tall
065-098	Plastizote, 1/4", 15"
065-099	Plastizote, 1/4", 15" Tall
065-100	Plastizote, 1/4", 16"
065-101	Plastizote, 1/4", 16" Tall
065-102	Plastizote, 1/4", 17"
065-103	Plastizote, 1/4", 17" Tall
065-104	Plastizote, 1/4", 18"
065-105	Plastizote, 1/4", 18" Tall
065-106	Plastizote, 1/4", 19"
065-107	Plastizote, 1/4", 19" Tall
065-108	Plastizote, 1/4", 20"
065-109	Plastizote, 1/4", 20" Tall
065-126	Plastizote, 1/4", 10" (Decaf)
065-130	Plastizote, 1/4", 10" Tall (Decaf)
065-229	Plastizote, 1/4", 14" Deep
065-167	Plastizote, 1/4", 15" Deep
065-168	Plastizote, 1/4", 16" Deep
065-169	Plastizote, 1/4", 17" Deep
065-170	Plastizote, 1/4", 18" Deep
065-171	Plastizote, 1/4", 19" Deep
065-172	Plastizote, 1/4", 20" Deep
065-231	Plastizote, 1/4", 14" Tall Deep
065-174	Plastizote, 1/4", 15" Tall Deep
065-175	Plastizote, 1/4", 16" Tall Deep
065-176	Plastizote, 1/4", 17" Tall Deep
065-177	Plastizote, 1/4", 18" Tall Deep
065-178	Plastizote, 1/4", 19" Tall Deep
065-179	Plastizote, 1/4", 20" Tall Deep
065-232	Plastizote, 1/4", 14" Grande
065-181	Plastizote, 1/4", 15" Grande
065-182	Plastizote, 1/4", 16" Grande
065-183	Plastizote, 1/4", 17" Grande
065-184	Plastizote, 1/4", 18" Grande
065-185	Plastizote, 1/4", 19" Grande
065-186	Plastizote, 1/4", 20" Grande
065-233	Plastizote, 1/4", 14" Grande Deep
065-188	Plastizote, 1/4", 15" Grande Deep
065-189	Plastizote, 1/4", 16" Grande Deep
065-190	Plastizote, 1/4", 17" Grande Deep
065-191	Plastizote, 1/4", 18" Grande Deep
065-192	Plastizote, 1/4", 19" Grande Deep
065-193	Plastizote, 1/4", 20" Grande Deep

Java® Back Polyurethane Foam

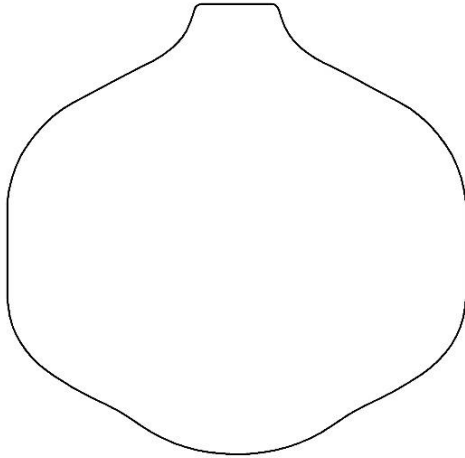


Part No.	Description
065-074	Polyurethane Foam, 1/2", 12" (Decaf)
065-075	Polyurethane Foam, 1/2", 12" Tall (Decaf)
065-076	Polyurethane Foam, 1/2", 13" (Decaf)
065-077	Polyurethane Foam, 1/2", 13" Tall (Decaf)
065-234	Polyurethane Foam, 1/2", 14"
065-236	Polyurethane Foam, 1/2", 14" Tall
065-080	Polyurethane Foam, 3/4", 15"
065-081	Polyurethane Foam, 3/4", 15" Tall
065-082	Polyurethane Foam, 3/4", 16"
065-083	Polyurethane Foam, 3/4", 16" Tall
065-084	Polyurethane Foam, 3/4", 17"
065-085	Polyurethane Foam, 3/4", 17" Tall
065-086	Polyurethane Foam, 3/4", 18"
065-087	Polyurethane Foam, 3/4", 18" Tall
065-088	Polyurethane Foam, 3/4", 19"
065-089	Polyurethane Foam, 3/4", 19" Tall
065-090	Polyurethane Foam, 3/4", 20"
065-091	Polyurethane Foam, 3/4", 20" Tall
065-124	Polyurethane Foam, 1/2", 10" (Decaf)
065-125	Polyurethane Foam, 1/2", 10" Tall (Decaf)
065-235	Polyurethane Foam, 1/2", 14" Deep
065-139	Polyurethane Foam, 3/4", 15" Deep
065-140	Polyurethane Foam, 3/4", 16" Deep
065-141	Polyurethane Foam, 3/4", 17" Deep
065-142	Polyurethane Foam, 3/4", 18" Deep
065-143	Polyurethane Foam, 3/4", 19" Deep
065-144	Polyurethane Foam, 3/4", 20" Deep
065-237	Polyurethane Foam, 1/2", 14" Tall Deep
065-146	Polyurethane Foam, 3/4", 15" Tall Deep
065-147	Polyurethane Foam, 3/4", 16" Tall Deep
065-148	Polyurethane Foam, 3/4", 17" Tall Deep
065-149	Polyurethane Foam, 3/4", 18" Tall Deep
065-150	Polyurethane Foam, 3/4", 19" Tall Deep
065-151	Polyurethane Foam, 3/4", 20" Tall Deep
065-238	Polyurethane Foam, 1/2", 14" Grande
065-153	Polyurethane Foam, 3/4", 15" Grande
065-154	Polyurethane Foam, 3/4", 16" Grande
065-155	Polyurethane Foam, 3/4", 17" Grande
065-156	Polyurethane Foam, 3/4", 18" Grande
065-157	Polyurethane Foam, 3/4", 19" Grande
065-158	Polyurethane Foam, 3/4", 20" Grande
065-239	Polyurethane Foam, 1/2", 14" Grande Deep
065-160	Polyurethane Foam, 3/4", 15" Grande Deep
065-161	Polyurethane Foam, 3/4", 16" Grande Deep
065-162	Polyurethane Foam, 3/4", 17" Grande Deep
065-163	Polyurethane Foam, 3/4", 18" Grande Deep
065-164	Polyurethane Foam, 3/4", 19" Grande Deep
065-165	Polyurethane Foam, 3/4", 20" Grande Deep

Java® Back Cover

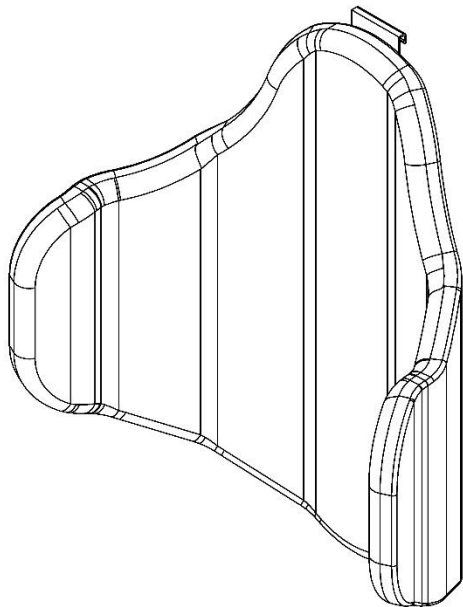


Part No.	Description
CJB-1010D	Java Back, Extra Cover, 10x10, Decaf, Regular
CJB-1013D	Java Back, Extra Cover, 10x13, Decaf, Tall
CJB-1212	Java Back Extra Cover, 12x12, Regular
CJB-1215	Java Back Extra Cover, 12x15, Tall
CJB-1312	Java Back Extra Cover, 13x12, Regular
CJB-1315	Java Back Extra Cover, 13x15, Tall
CJB-1412	Java Back Extra Cover, 14x12, Regular
CJB-1412D	Java Back, Extra Cover, 14x12, Decaf, Regular
CJB-1412DP	Java Back Extra Cover, 14x12, Deep,
CJB-1415DP	Java Back Extra Cover, 14x15T, Deep, Tall
CJB-1415D	Java Back, Extra Cover, 14x15, Decaf, Tall
CJB-1415	Java Back Extra Cover, 14x15, Tall
CJB-1419	Java Back Extra Cover, 14x19, Grande
CJB-1419DP	Java Back Extra Cover, 14x19, Grande Deep
CJB-1514	Java Back Extra Cover, 15x14, Regular
CJB-1514DP	Java Back Extra Cover, 15x14, Deep,
CJB-1518DP	Java Back Extra Cover, 15x18, Deep, Tall
CJB-1518	Java Back Extra Cover, 15x18, Tall
CJB-1522	Java Back Extra Cover, 15x22, Grande
CJB-1522DP	Java Back Extra Cover, 15x22, Grande Deep
CJB-1614	Java Back Extra Cover, 16x14, Regular
CJB-1614DP	Java Back Extra Cover, 16x14, Deep,
CJB-1618DP	Java Back Extra Cover, 16x18, Deep, Tall
CJB-1618	Java Back Extra Cover, 16x18, Tall
CJB-1622	Java Back Extra Cover, 16x22, Grande
CJB-1622DP	Java Back Extra Cover, 16x22, Grande Deep
CJB-1714	Java Back Extra Cover, 17x14, Regular
CJB-1714DP	Java Back Extra Cover, 17x14, Deep,
CJB-1718DP	Java Back Extra Cover, 17x18, Deep, Tall
CJB-1718	Java Back Extra Cover, 17x18, Tall
CJB-1722	Java Back Extra Cover, 17x22, Grande
CJB-1722DP	Java Back Extra Cover, 17x22, Grande Deep
CJB-1814	Java Back Extra Cover, 18x14, Regular
CJB-1814DP	Java Back Extra Cover, 18x14, Deep,
CJB-1818DP	Java Back Extra Cover, 18x18, Deep, Tall
CJB-1818	Java Back Extra Cover, 18x18, Tall
CJB-1822	Java Back Extra Cover, 18x22, Grande
CJB-1822DP	Java Back Extra Cover, 18x22, Grande Deep
CJB-1914	Java Back Extra Cover, 19x14, Regular
CJB-1914DP	Java Back Extra Cover, 19x14, Deep,
CJB-1918DP	Java Back Extra Cover, 19x18, Deep, Tall
CJB-1918	Java Back Extra Cover, 19x18, Tall
CJB-1922	Java Back Extra Cover, 19x22, Grande
CJB-1922DP	Java Back Extra Cover, 19x22, Grande Deep
CJB-2014	Java Back Extra Cover, 20x14, Regular
CJB-2014DP	Java Back Extra Cover, 20x14, Deep,
CJB-2018DP	Java Back Extra Cover, 20x18, Deep, Tall
CJB-2018	Java Back Extra Cover, 20x18, Tall
CJB-2022	Java Back Extra Cover, 20x22, Grande
CJB-2022DP	Java Back Extra Cover, 20x22, Grande Deep

Java® Back Fixed Lateral Assembly

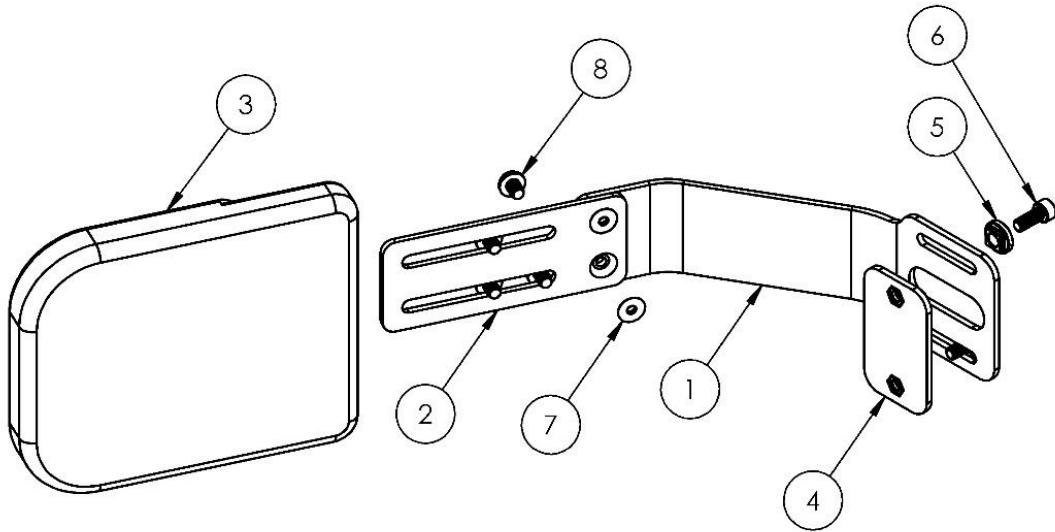


Item	Part No.	Description
1	See Chart	Java Lateral Bracket, Fixed
2	See Chart	Lateral Pad Bracket
3	See Chart	Lateral Pad
4	025-153	M6 Nut Plate
5	025-313	M6 Slot Nut (Non-Threaded)
6	015-220	M6 x 15mm Socket Head Cap Screw
7	015-399	Flat Head Undercut Cap Screw 1/4"-20 x 0.25
8	015-397	Button Head Cap Screw with Washer 1/4"-20 x 0.5


Java[®] Back Fixed Lateral Assembly

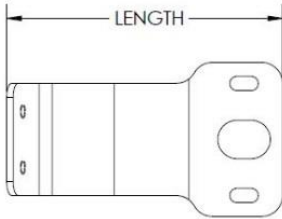


Part No.	Description	Length
025-224	Java Lateral Bracket, Fixed, 10" Decaf	4.91"
025-225	Java Lateral Bracket, Fixed, 12"- 14" Decaf	5.87"
025-226	Java Lateral Bracket, Fixed, 15"- 20"	7.37"

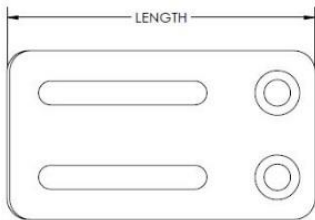


Part No.	Description	Length
025-232	Lateral Pad Bracket, Curved, Short	3.66"
025-233	Lateral Pad Bracket, Curved, Long	4.66"

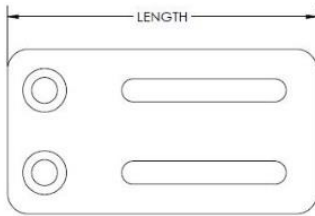

Part No.	Description	Length
025-230	Lateral Pad Bracket, Flat, Short	3.76"
025-231	Lateral Pad Bracket, Flat, Long	4.76"

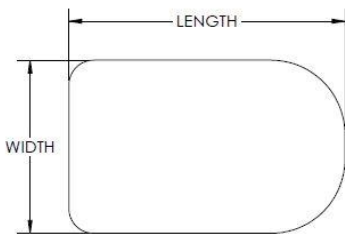


Part No.	Description	Width	Length
065-210	Lateral Pad, Flat, 3" x 5"	3.5"	5.6"
065-211	Lateral Pad, Flat, 4" x 6"	4.5"	6.6"
065-212	Lateral Pad, Flat, 5" x 6"	5.5"	6.6"

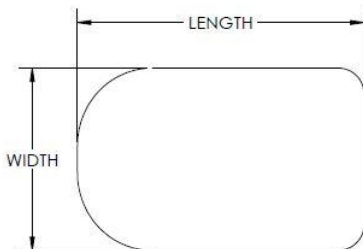


Part No.	Description	Width	Length
065-213	Lateral Pad, Curved, 3" x 5"	3.5"	5.6"
065-214	Lateral Pad, Curved, 4" x 6"	4.5"	6.6"
065-215	Lateral Pad, Curved, 5" x 6"	5.5"	6.6"

Java® Back Swing Away Lateral Assembly, Right

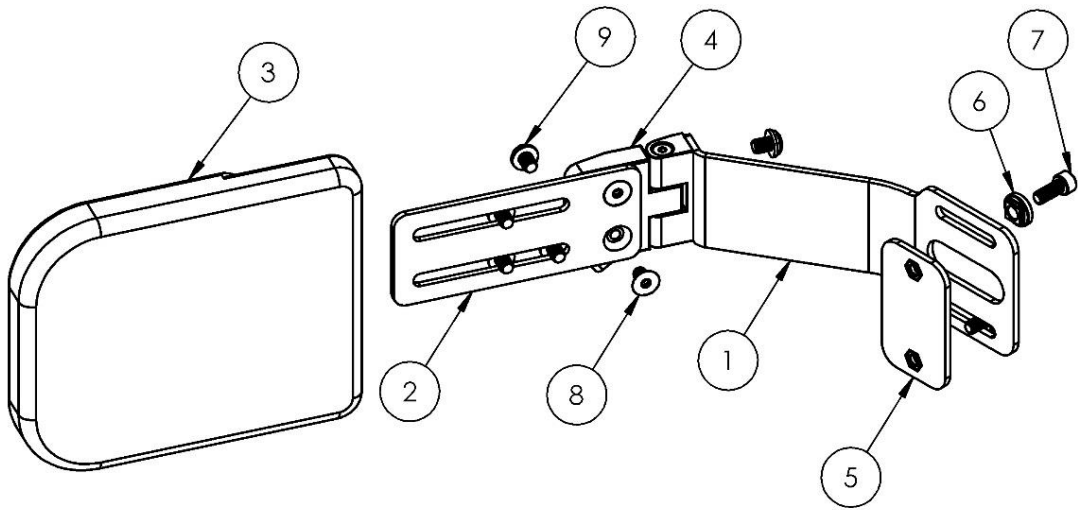


Item	Part No.	Description
1	See Chart	Java Lateral Bracket, Swing Away
2	See Chart	Lateral Pad Bracket
3	See Chart	Lateral Pad
4	6000-389	Swing Away Lateral Hinge, Right
5	025-153	M6 Nut Plate
6	025-313	M6 Slot Nut (Non-Threaded)
7	015-220	M6 x 15mm Socket Head Cap Screw
8	015-398	Flat Head Undercut Cap Screw 1/4"-20 x 0.375
9	015-397	Button Head Cap Screw with Washer 1/4"-20 x 0.5


Java® Back Swing Away Lateral Assembly, Right

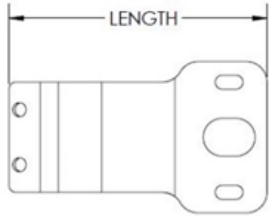


Part No.	Description	Length
025-227	Java Lateral Bracket, Swing Away, 10" Decaf	4.67"
025-228	Java Lateral Bracket, Swing Away, 12"-14" Decaf	5.63"
025-229	Java Lateral Bracket, Swing Away, 15"-20"	6.95"

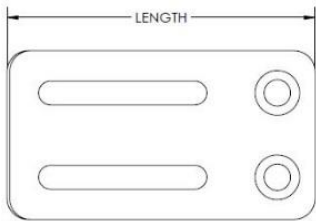


Part No.	Description	Length
025-232	Lateral Pad Bracket, Curved, Short	3.66"
025-233	Lateral Pad Bracket, Curved, Long	4.66"

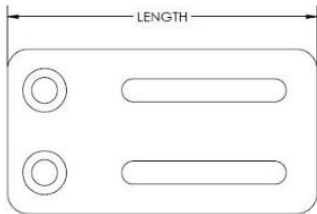

Part No.	Description	Length
025-230	Lateral Pad Bracket, Flat, Short	3.76"
025-231	Lateral Pad Bracket, Flat, Long	4.76"

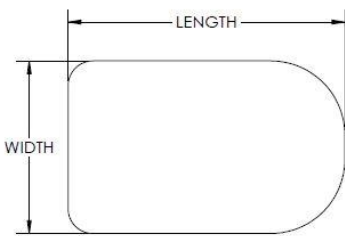


Part No.	Description	Width	Length
065-210	Lateral Pad, Flat, 3" x 5"	3.5"	5.6"
065-211	Lateral Pad, Flat, 4" x 6"	4.5"	6.6"
065-212	Lateral Pad, Flat, 5" x 6"	5.5"	6.6"

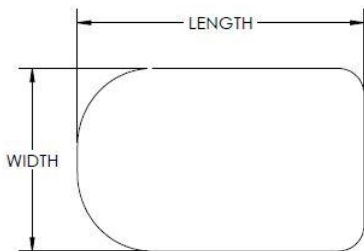


Part No.	Description	Width	Length
065-213	Lateral Pad, Curved, 3" x 5"	3.5"	5.6"
065-214	Lateral Pad, Curved, 4" x 6"	4.5"	6.6"
065-215	Lateral Pad, Curved, 5" x 6"	5.5"	6.6"

Java® Back Swing Away Lateral Assembly, Left

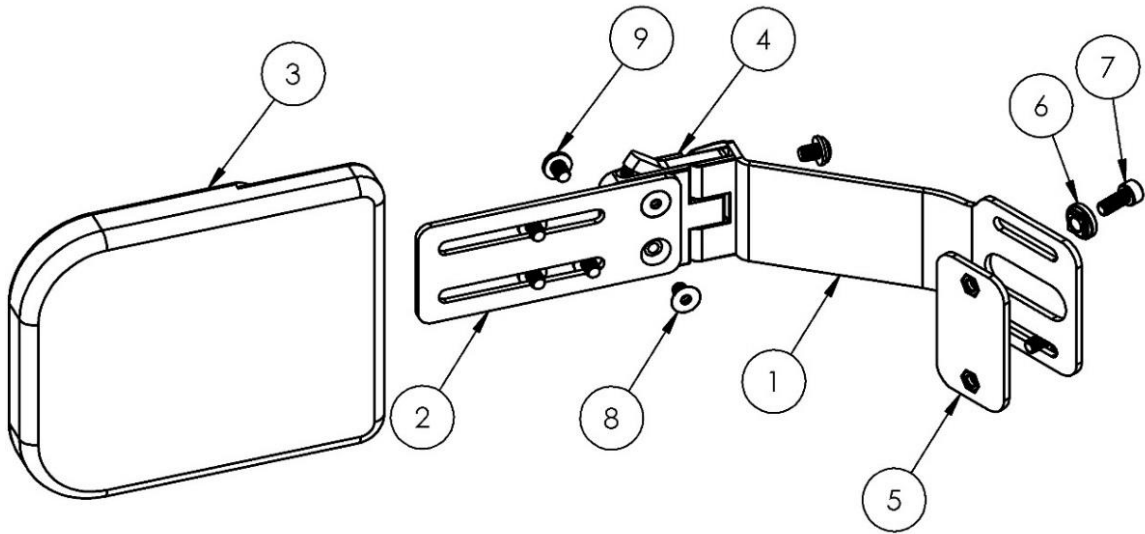


Item	Part No.	Description
1	See Chart	Java Lateral Bracket, Swing Away
2	See Chart	Lateral Pad Bracket
3	See Chart	Lateral Pad
4	6000-390	Swing Away Lateral Hinge, Left
5	025-153	M6 Nut Plate
6	025-313	M6 Slot Nut (Non-Threaded)
7	015-220	M6 x 15mm Socket Head Cap Screw
8	015-398	Flat Head Undercut Cap Screw 1/4"-20 x 0.375
9	015-397	Button Head Cap Screw with Washer 1/4"-20 x 0.5


Java[®] Back Swing Away Lateral Assembly, Left

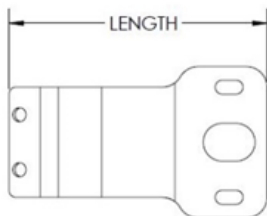


Part No.	Description	Length
025-227	Java Lateral Bracket, Swing Away, 10" Decaf	4.67"
025-228	Java Lateral Bracket, Swing Away, 12"-14" Decaf	5.63"
025-229	Java Lateral Bracket, Swing Away, 15"-20"	6.95"

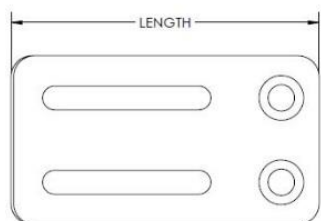


Part No.	Description	Length
025-232	Lateral Pad Bracket, Curved, Short	3.66"
025-233	Lateral Pad Bracket, Curved, Long	4.66"

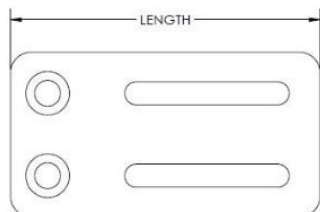

Part No.	Description	Length
025-230	Lateral Pad Bracket, Flat, Short	3.76"
025-231	Lateral Pad Bracket, Flat, Long	4.76"

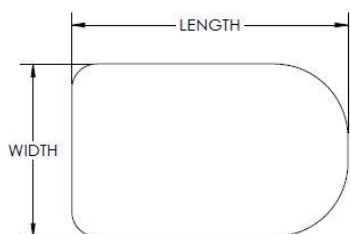


Part No.	Description	Width	Length
065-210	Lateral Pad, Flat, 3" x 5"	3.5"	5.6"
065-211	Lateral Pad, Flat, 4" x 6"	4.5"	6.6"
065-212	Lateral Pad, Flat, 5" x 6"	5.5"	6.6"

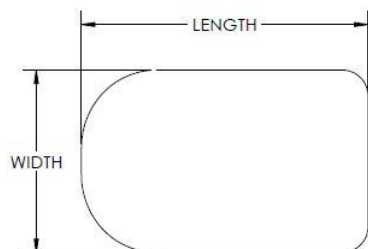

Part No.	Description	Width	Length
065-213	Lateral Pad, Curved, 3" x 5"	3.5"	5.6"
065-214	Lateral Pad, Curved, 4" x 6"	4.5"	6.6"
065-215	Lateral Pad, Curved, 5" x 6"	5.5"	6.6"


Ride[®] Java[®] Cushion Parts

Java® Cushion Assembly

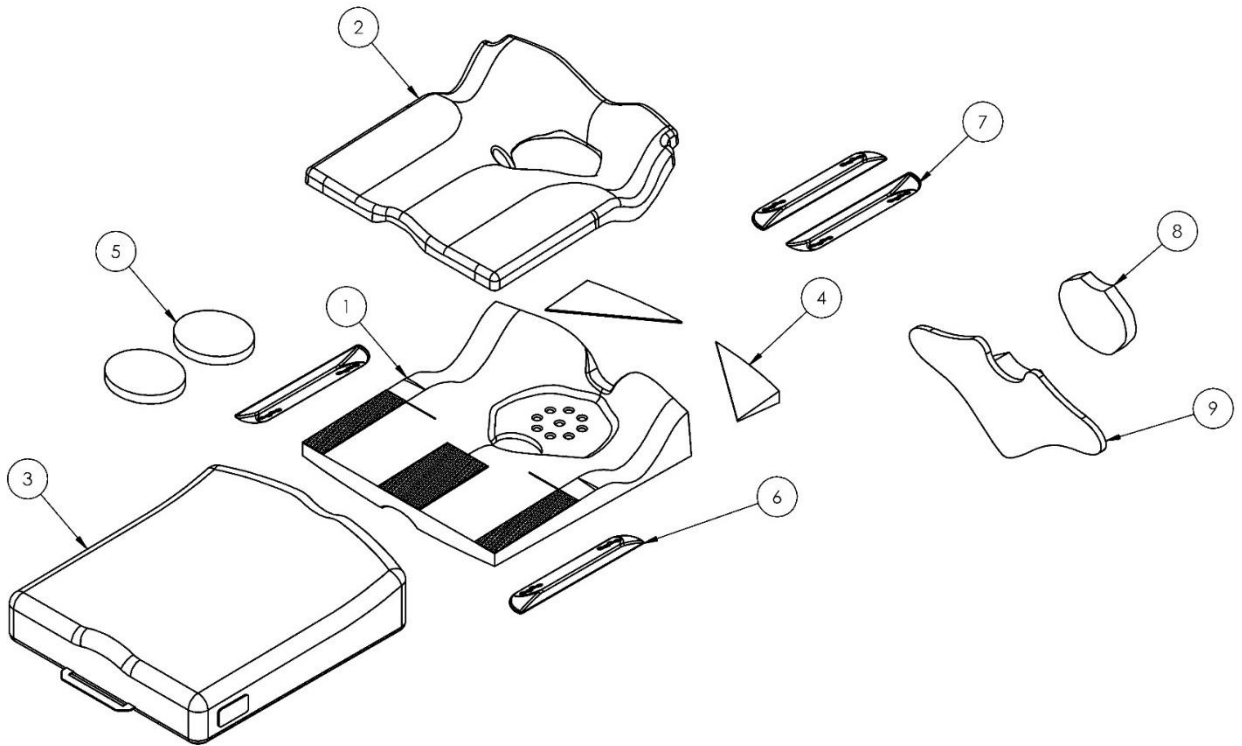


Item	Part No.	Description
1	See Chart	Java Cushion Base
2	See Chart	Java Cushion Top Foam Layer
3	JCE-_-_-_-	Java Cushion Extra Cover (Insert Cushion Size)
4	JC-WK	Ride CAM Wedge Kit (Pair)
5	6000-290	Java Cushion 28 Gauge (Pair)
6	JC-LTS	Lateral Thigh Support Wedge (Pair)
7	JC-MTS	Medial Thigh Support Wedge (Pair)
8	065-202	Well Insert, Bottom
9	065-201	Well Insert, Top

Java® Cushion Top Foam

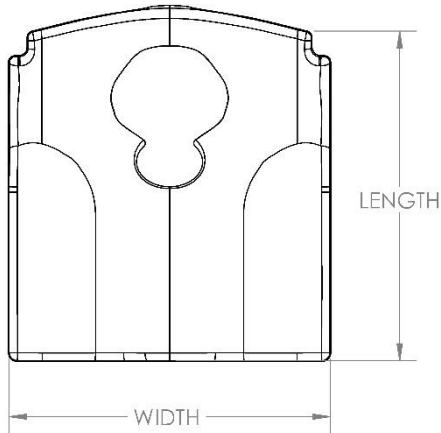

Part No.	Description		
6000-524	Replacement Top Foam Assy, 12" x 12"	12"	11.5"
6000-525	Replacement Top Foam Assy, 12" x 13"	12"	12.5"
6000-526	Replacement Top Foam Assy, 12" x 14"	12"	13.5"
6000-527	Replacement Top Foam Assy, 12" x 15"	12"	14.5"
6000-528	Replacement Top Foam Assy, 12" x 16"	12"	15.5"
6000-535	Replacement Top Foam Assy, 13" x 12"	13"	11.5"
6000-536	Replacement Top Foam Assy, 13" x 13"	13"	12.5"
6000-537	Replacement Top Foam Assy, 13" x 14"	13"	13.5"
6000-538	Replacement Top Foam Assy, 13" x 15"	13"	14.5"
6000-539	Replacement Top Foam Assy, 13" x 16"	13"	15.5"
6000-546	Replacement Top Foam Assy, 14" x 12"	14"	11.5"
6000-547	Replacement Top Foam Assy, 14" x 13"	14"	12.5"
6000-548	Replacement Top Foam Assy, 14" x 14"	14"	12.5"
6000-549	Replacement Top Foam Assy, 14" x 15"	14"	12.5"
6000-550	Replacement Top Foam Assy, 14" x 16"	14"	12.5"
6000-551	Replacement Top Foam Assy, 14" x 17"	14"	12.5"
6000-552	Replacement Top Foam Assy, 14" x 18"	14"	12.5"
6000-559	Replacement Top Foam Assy, 15" x 14"	15"	13.5"
6000-560	Replacement Top Foam Assy, 15" x 15"	15"	14.5"
6000-561	Replacement Top Foam Assy, 15" x 16"	15"	15.5"
6000-562	Replacement Top Foam Assy, 15" x 17"	15"	16.5"
6000-563	Replacement Top Foam Assy, 15" x 18"	15"	17.5"
6000-564	Replacement Top Foam Assy, 15" x 19"	15"	18.5"
6000-565	Replacement Top Foam Assy, 15" x 20"	15"	19.5"
6000-570	Replacement Top Foam Assy, 16" x 14"	16"	13.5"
6000-571	Replacement Top Foam Assy, 16" x 15"	16"	14.5"
6000-572	Replacement Top Foam Assy, 16" x 16"	16"	15.5"
6000-573	Replacement Top Foam Assy, 16" x 17"	16"	16.5"
6000-574	Replacement Top Foam Assy, 16" x 18"	16"	17.5"
6000-575	Replacement Top Foam Assy, 16" x 19"	16"	18.5"
6000-576	Replacement Top Foam Assy, 16" x 20"	16"	19.5"
6000-581	Replacement Top Foam Assy, 17" x 14"	17"	13.5"
6000-582	Replacement Top Foam Assy, 17" x 15"	17"	14.5"
6000-583	Replacement Top Foam Assy, 17" x 16"	17"	15.5"
6000-584	Replacement Top Foam Assy, 17" x 17"	17"	16.5"
6000-585	Replacement Top Foam Assy, 17" x 18"	17"	17.5"
6000-586	Replacement Top Foam Assy, 17" x 19"	17"	18.5"
6000-587	Replacement Top Foam Assy, 17" x 20"	17"	19.5"
6000-592	Replacement Top Foam Assy, 18" x 14"	18"	13.5"
6000-593	Replacement Top Foam Assy, 18" x 15"	18"	14.5"
6000-594	Replacement Top Foam Assy, 18" x 16"	18"	15.5"
6000-595	Replacement Top Foam Assy, 18" x 17"	18"	16.5"
6000-596	Replacement Top Foam Assy, 18" x 18"	18"	17.5"
6000-597	Replacement Top Foam Assy, 18" x 19"	18"	18.5"
6000-598	Replacement Top Foam Assy, 18" x 20"	18"	19.5"
6000-603	Replacement Top Foam Assy, 19" x 14"	19"	13.5"
6000-604	Replacement Top Foam Assy, 19" x 15"	19"	14.5"
6000-605	Replacement Top Foam Assy, 19" x 16"	19"	15.5"
6000-606	Replacement Top Foam Assy, 19" x 17"	19"	16.5"
6000-607	Replacement Top Foam Assy, 19" x 18"	19"	17.5"
6000-608	Replacement Top Foam Assy, 19" x 19"	19"	18.5"
6000-609	Replacement Top Foam Assy, 19" x 20"	19"	19.5"
6000-614	Replacement Top Foam Assy, 20" x 14"	20"	13.5"
6000-615	Replacement Top Foam Assy, 20" x 15"	20"	14.5"
6000-616	Replacement Top Foam Assy, 20" x 16"	20"	15.5"
6000-617	Replacement Top Foam Assy, 20" x 17"	20"	16.5"
6000-618	Replacement Top Foam Assy, 20" x 18"	20"	17.5"
6000-619	Replacement Top Foam Assy, 20" x 19"	20"	18.5"
6000-620	Replacement Top Foam Assy, 20" x 20"	20"	19.5"

Java® Cushion Base

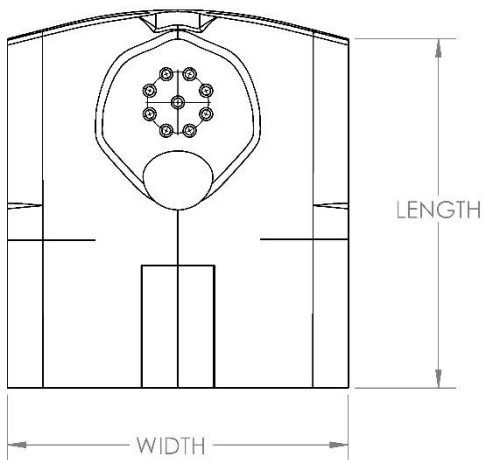

Part No.	Description		
6000-425	Replacement Base Assy, 12" x 12"	12"	11.5"
6000-426	Replacement Base Assy, 12" x 13"	12"	12.5"
6000-427	Replacement Base Assy, 12" x 14"	12"	13.5"
6000-428	Replacement Base Assy, 12" x 15"	12"	14.5"
6000-429	Replacement Base Assy, 12" x 16"	12"	15.5"
6000-436	Replacement Base Assy, 13" x 12"	13"	11.5"
6000-437	Replacement Base Assy, 13" x 13"	13"	12.5"
6000-438	Replacement Base Assy, 13" x 14"	13"	13.5"
6000-439	Replacement Base Assy, 13" x 15"	13"	14.5"
6000-440	Replacement Base Assy, 13" x 16"	13"	15.5"
6000-447	Replacement Base Assy, 14" x 12"	14"	11.5"
6000-448	Replacement Base Assy, 14" x 13"	14"	12.5"
6000-449	Replacement Base Assy, 14" x 14"	14"	12.5"
6000-450	Replacement Base Assy, 14" x 15"	14"	12.5"
6000-451	Replacement Base Assy, 14" x 16"	14"	12.5"
6000-452	Replacement Base Assy, 14" x 17"	14"	12.5"
6000-453	Replacement Base Assy, 14" x 18"	14"	12.5"
6000-460	Replacement Base Assy, 15" x 14"	15"	13.5"
6000-461	Replacement Base Assy, 15" x 15"	15"	14.5"
6000-462	Replacement Base Assy, 15" x 16"	15"	15.5"
6000-463	Replacement Base Assy, 15" x 17"	15"	16.5"
6000-464	Replacement Base Assy, 15" x 18"	15"	17.5"
6000-465	Replacement Base Assy, 15" x 19"	15"	18.5"
6000-466	Replacement Base Assy, 15" x 20"	15"	19.5"
6000-471	Replacement Base Assy, 16" x 14"	16"	13.5"
6000-472	Replacement Base Assy, 16" x 15"	16"	14.5"
6000-473	Replacement Base Assy, 16" x 16"	16"	15.5"
6000-474	Replacement Base Assy, 16" x 17"	16"	16.5"
6000-475	Replacement Base Assy, 16" x 18"	16"	17.5"
6000-476	Replacement Base Assy, 16" x 19"	16"	18.5"
6000-477	Replacement Base Assy, 16" x 20"	16"	19.5"
6000-482	Replacement Base Assy, 17" x 14"	17"	13.5"
6000-483	Replacement Base Assy, 17" x 15"	17"	14.5"
6000-484	Replacement Base Assy, 17" x 16"	17"	15.5"
6000-485	Replacement Base Assy, 17" x 17"	17"	16.5"
6000-486	Replacement Base Assy, 17" x 18"	17"	17.5"
6000-487	Replacement Base Assy, 17" x 19"	17"	18.5"
6000-488	Replacement Base Assy, 17" x 20"	17"	19.5"
6000-493	Replacement Base Assy, 18" x 14"	18"	13.5"
6000-494	Replacement Base Assy, 18" x 15"	18"	14.5"
6000-495	Replacement Base Assy, 18" x 16"	18"	15.5"
6000-496	Replacement Base Assy, 18" x 17"	18"	16.5"
6000-497	Replacement Base Assy, 18" x 18"	18"	17.5"
6000-498	Replacement Base Assy, 18" x 19"	18"	18.5"
6000-499	Replacement Base Assy, 18" x 20"	18"	19.5"
6000-504	Replacement Base Assy, 19" x 14"	19"	13.5"
6000-505	Replacement Base Assy, 19" x 15"	19"	14.5"
6000-506	Replacement Base Assy, 19" x 16"	19"	15.5"
6000-507	Replacement Base Assy, 19" x 17"	19"	16.5"
6000-508	Replacement Base Assy, 19" x 18"	19"	17.5"
6000-509	Replacement Base Assy, 19" x 19"	19"	18.5"
6000-510	Replacement Base Assy, 19" x 20"	19"	19.5"
6000-515	Replacement Base Assy, 20" x 14"	20"	13.5"
6000-516	Replacement Base Assy, 20" x 15"	20"	14.5"
6000-517	Replacement Base Assy, 20" x 16"	20"	15.5"
6000-518	Replacement Base Assy, 20" x 17"	20"	16.5"
6000-519	Replacement Base Assy, 20" x 18"	20"	17.5"
6000-520	Replacement Base Assy, 20" x 19"	20"	18.5"
6000-521	Replacement Base Assy, 20" x 20"	20"	19.5"