

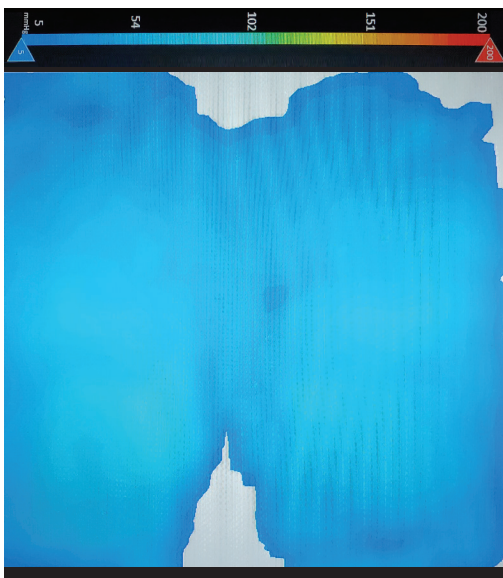


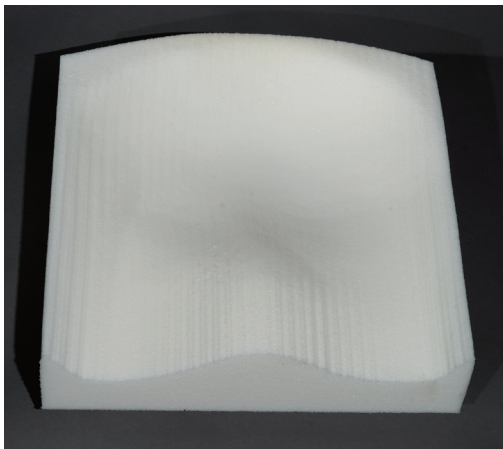




# Ride® Custom AccuSoft™ Cushion: Full Contact vs Ride Contour Option

## Full Contact option ▼

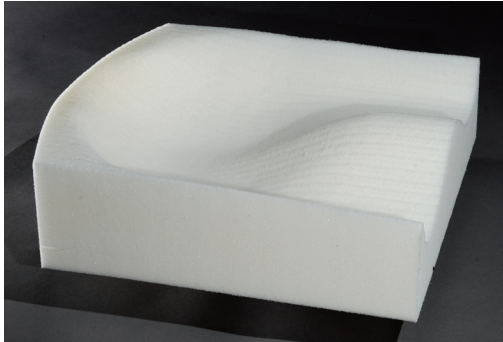

## Ride Contour option ▼

(made from same shape as seen at left, except with Ride Contour option applied)

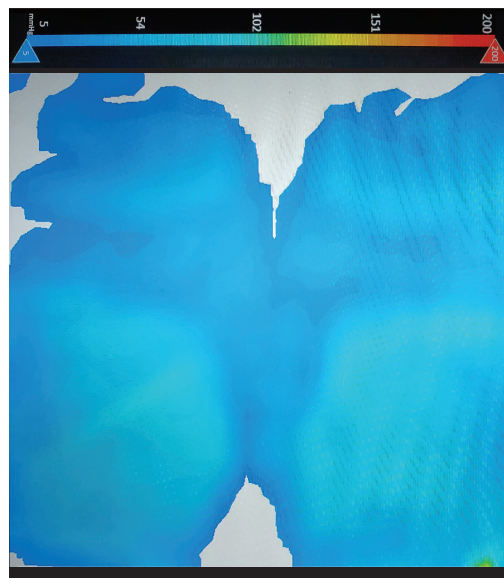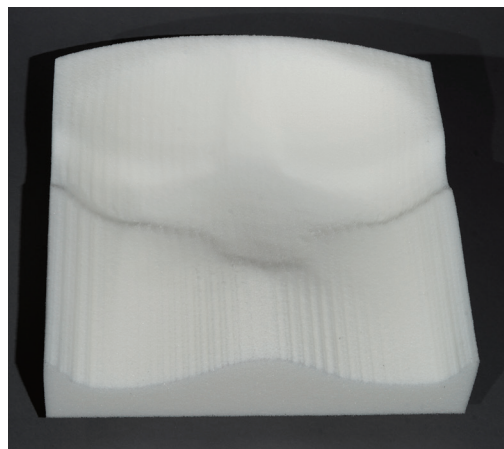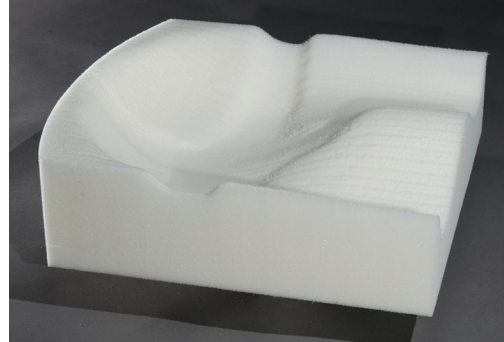

### What does selecting the Ride Contour option mean?

The Ride Contour option carves a recess 3/8 inch deep into the surface of the cushion underneath the high risk bony areas.

### The Ride Contour option does not create offloading under the ischial tuberosities.

Rather, it creates a loading bias underneath the proximal thighs and posterior-lateral buttocks, which then reduces forces underneath the ischial tuberosities (as shown in pressure map image, below right).

The Ride Contour option is not available when ordering extra soft foam.

NOTE: The Ride Custom AccuSoft Cushion is not a fully off-loading cushion. For orthotic off-loading, consider using the Java® Cushion or Ride Custom 2 Cushion for the ultimate in skin safety and postural support.

**Ride Designs®**  
a branch of Aspen Seating, LLC

toll-free: 866.781.1633  
phone: 303.781.1633  
[www.ridedesigns.com](http://www.ridedesigns.com)