


► **Ride Designs®**
a branch of Aspen Seating, LLC
8100 SouthPark Way, C400
Littleton, Colorado 80120 USA
toll-free: 866.781.1633
phone: 303.781.1633
www.ridedesigns.com

FlexLoc® Change from Quick Release to Fixed Pin

Instructions:

1. Remove FlexLoc® from receiver assembly.
2. Using a 2.5mm Allen wrench, remove the 3ea set screws from the quick release lever. Remove the lever.
3. Using a 5/8" wrench remove the adjustment nut on the top of the quick release pin.
4. Using a 7/16" wrench (not included) unscrew the quick release pin from the FlexLoc assembly.
5. Place FlexLoc assembly in receiver assembly.
6. Slide fixed pin through the FlexLoc assembly and the receiver assembly.
7. Using the 4mm Allen wrench and the flats of the 5/8" wrench secure the fixed pin, wave washer and flat washer together (do not pinch wave washer between fixed pin and flat washer).

1, 2

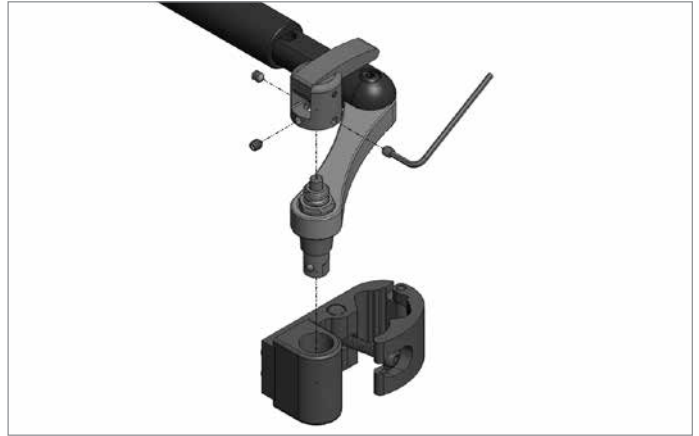

3



4



5



6



7

